
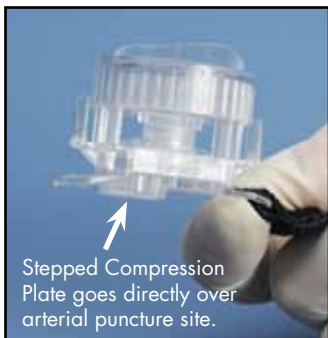


# Finale® Radial Compression Device

**PLEASE CONSULT INSTRUCTIONS FOR USE FOR ADDITIONAL INFORMATION.**

**STEP 1.** To prepare the Finale™ device for use, squeeze tabs and rotate dial counter-clockwise until it stops.

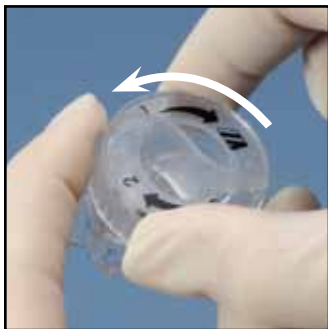

US & Foreign Patents Pending.

**STEP 2.** Place the device on patient. The compression plate is placed directly over the arterial puncture site. The sheath will be under the compression plate. Loop VELCRO® brand strap through slot; attach and tighten appropriately. Strap should be snug. **DO NOT OVER TIGHTEN STRAP.**



VELCRO® is a registered trademark of Velcro Industries B.V.

**STEP 3.** Remove sheath while rotating dial clockwise to achieve hemostasis. To avoid over-tightening the device, decrease compression until slight bleeding is observed; slowly increase compression until bleeding stops.



**Warning – Do not over-tighten device.**


Catalog Number	Band Length
<b>FCD100</b>	10" (25.4cm)
<b>FCD200</b>	12" (30.5cm)

**STEP 4.** Initial compression should be maintained for a minimum of 30 minutes; however, some patients may require longer initial compression times. Continue to check for bleeding at regular intervals; if bleeding occurs, adjust compression by turning dial clockwise. **Note – Use minimum amount of pressure necessary to achieve hemostasis.**



**Warning – Patient should not be left unattended while device is in place.**

**STEP 5.** To decrease compression, squeeze release tabs and rotate dial counter-clockwise to desired compression. Compression may be reduced every 15-30 minutes; however, some patients may require longer time intervals. **Note - If bleeding is observed from access site, increase compression until bleeding stops.**

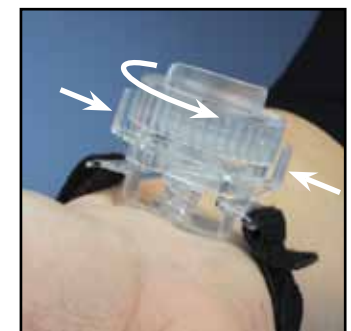

**STEP 6.** Once hemostasis is achieved, remove device and discard.

**MERITMEDICAL™ RADIAL APPROACH IT'S TIME FOR OPTIONS™**

Merit Medical Systems, Inc. • 1600 West Merit Parkway • South Jordan, Utah 84095 • 1-801-253-1600 • 1-800-35-MERIT

Merit Medical EUROPE, MIDDLE EAST AND AFRICA (EMEA) • Amerikalaan 42, 6199 AE Maastricht-Airport • The Netherlands • Tel: +31 43 358 82 40

Free phone for specific country: Austria 0800 295374 • Belgium 0800 729 06 (Dutch) 0800 731 72 (French) • Denmark 80 88 00 24 • France 0800 91 60 30 • Germany 0800 182 08 71

Republic of Ireland 1800 553 163 • Luxembourg 8002 25 22 • Netherlands 0800 022 8184 • Sweden 020 79 24 45 • UK 0800 973 115 • Finland 00800 6374 8633

www.merit.com

401926002/A ID 060911