

Clo-Sur^{PLUS} P.A.D.TM

PATIENT INSTRUCTIONS

Recommended instructions for removing Clo-Sur PLUS P.A.D.

- 1** After 24 hours, remove dressing: Wet Clo-Sur PLUS P.A.D with water and allow it to dissolve and remove slowly.
- 2** Observe treatment area for any signs of infection, including drainage, swelling, redness, increased pain, etc. Watch for any signs of hematoma development, which will be a hard painful bruise at the treatment area.
- 3** If a hematoma or bleeding develops, hold pressure at the site, call your physician immediately or go to the emergency room.
- 4** Call your physician's office if you have any problems or concerns.

NOTE : These Instructions are to be used as guidelines. Please follow the instructions given to you by your doctor and/or health care provider.

Distributed by



Manufactured by



Scion Cardio-Vascular, Inc.

14256 S.W. 119th Avenue, Miami, FL 33186 • 305.259.8880

www.scioncv.com