



PERITONEAL

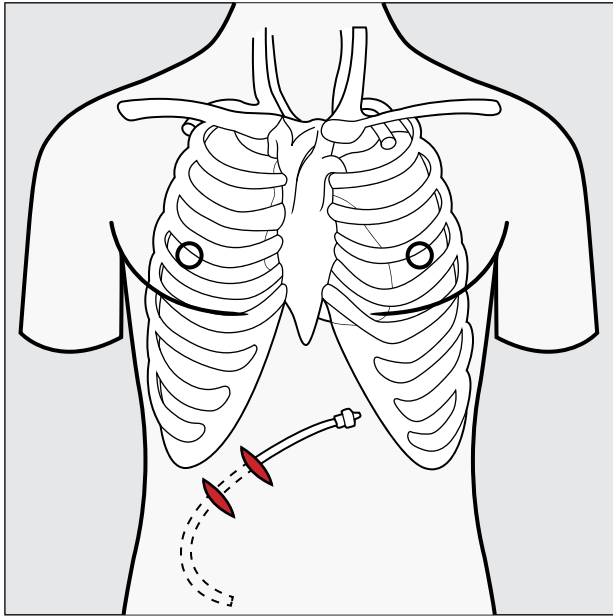

Make an incision at the desired catheter insertion site. Make another incision *superior* and medial to the insertion site at a distance selected for tunnel length (generally 5-8 cm).

PLEURAL

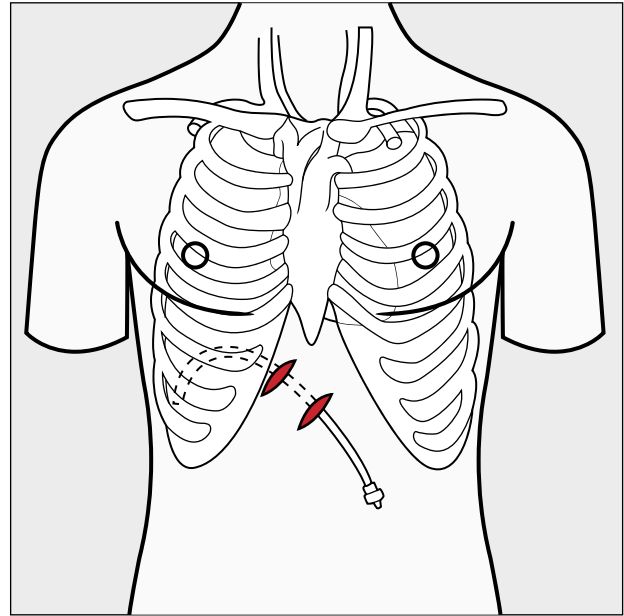

Make an incision at the desired catheter insertion site. Make another incision *inferior* and medial to the insertion site at a distance selected for tunnel length (generally 5-8 cm).

The techniques presented herein are for informational purposes only and the decision of which technique to use in a particular surgical application should be made by the physician based upon individual facts and circumstances of the patient and previous surgical experience.

Before using refer to Instructions for Use for indications, contraindications, warnings, precautions, and directions for use.



Understand. Innovate. Deliver.™

[merit.com](http://merit.com) | [myaspira.com](http://myaspira.com)

Merit Medical Systems, Inc.  
1600 West Merit Parkway  
South Jordan, Utah 84095  
1.801.253.1600  
1.800.35.MERIT

Merit Medical Europe, Middle  
East, & Africa (EMEA)  
Amerikalaan 42, 6199 AE  
Maastricht-Airport  
The Netherlands  
+31 43 358 82 22

Merit Medical Ireland Ltd.  
Parkmore Business Park West  
Galway, Ireland  
+353 (0) 91 703 733