

# PRESS FOR BETTER SAFETY



## BiopSafe®

Biopsy Sample System

Formalin capsuled lid for safe  
and fast biospy handling.

## IT'S ABOUT REMOVING A RECOGNIZED RISK...

One of the most common problems with biopsy handling is the risk of being exposed to formalin and formaldehyde either through touch or inhalation.

The IARC, which is the Cancer Agency of the World Health Organization, classifies formaldehyde as a category **1b carcinogen**.\*

Suited to a wide range of biopsy procedures, BiopSafe offers the same quality sample fixation you're used to, but with additional safety.



## PRODUCT BENEFITS:

- Safe sample handling with no exposure to formalin or formaldehyde via touch or inhalation
- Ready in seconds
- No sensory irritation
- Optimal formalin volumes minimizes waste and expensive disposal costs

## THE SECRET IS IN THE LID

The Biosafe system consists of a small, easy to use container (vial) with formalin capsuled in the lid. The specimen is placed into the container, and the lid is screwed in place. Depressing the top of the lid allows the formalin to flow down and cover the sample. No formalin is released outside the container in liquid or vapor form.



## STEP BY STEP



1 Remove the blue lid.



2 The specimen is placed in the vial.



3 The lid is screwed on.



4 The formalin is released with a press of the thumb.



5 The formalin and the biopsy is mixed.



6 The biopsy is ready for transportation.

# BiopSafe®

Biopsy Sample System

BiopSafe makes everyday life easier and safer for nurses, physicians, and laboratory technicians – both when working at the hospital or the clinic.

## ORDERING INFORMATION

Catalog Number	Size
3178-200-00	20ml
3178-600-00	60ml



Before using, refer to Instructions for Use for indications, contraindications, warnings, precautions and directions for use.



Understand. Innovate. Deliver.™

Merit Medical Systems, Inc.  
1600 West Merit Parkway  
South Jordan, Utah 84095  
1.801.253.1600  
1.800.35.MERIT

Merit Medical Europe, Middle East, & Africa (EMEA)  
Amerikalaan 42, 6199 AE Maastricht-Airport  
The Netherlands  
+31 43 358 82 22

Merit Medical Ireland Ltd.  
Parkmore Business Park West  
Galway, Ireland  
+353 (0) 91 703 733

[merit.com](http://merit.com)

Australia  
+61 (0) 3 8373 4943

Austria  
0800 295 374

Belgium  
0800 72 906 (Dutch)  
0800 73 172 (Français)

Brazil  
+55 11 4561 - 0062

Canada  
1.800.364.4370

China  
+86-10 8561 0788

Denmark  
80 88 00 24

Finland  
0800 770 586

France  
0800 91 60 30

Germany  
0800 182 0871

Hong Kong  
+852-2207 6888

India  
+91-80-41223376

Ireland (Republic)  
1800 553 163

Italy  
800 897 005

Luxembourg  
8002 25 22

Netherlands  
0800 022 81 84

Norway  
800 11629

Portugal  
308 801 034

Russia  
+7 495 221 89 02

Singapore  
+65 67536311

South Korea  
+8210 2205 0707

Spain  
+34 91 1238406

Sweden  
020 792 445

Switzerland  
(Deutsch)  
+41 22 518 02 30

(Français)  
+41 22 518 02 52

(Italiano)  
+41 22 518 00 35

UK  
0800 973 115