Rad Trac™

Arm Board Support Base

Base de soutien du Panneau à bras, Base di supporto del poggibraccia, Unterstützungsgrundlage für das Arm Brett, Base para apoyo de placa de brazo, Base de suporte para apoio de braço, Basisplaat voor armondersteuning, Stödbas för armbord, Underlag til armbræt, Βάση στήριξης σανίδας βραχίονα, Kol Tahtasi Destegi, Держатель доски для руки

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

The Merit Medical Rad Trac™ is made of rigid PVC and designed to be used in conjunction with the Merit Medical Rad Board® (sold separately). The Rad Trac facilitates placement of the Rad Board with a patient already on the procedure table (Figure 1).

INDICATIONS FOR USE

The Rad Trac is designed to allow easy placement and removal of Rad Board while the patient is on the procedure table.

WARNINGS

- The legs of the Rad Trac may cause a light artefact when imaged under fluoroscopy. The artifact is hazy and will not impede clinical visualization for radial procedures.
- DO NOT sit, stand or place heavy objects on the Rad Trac.
- DO NOT drop Rad Trac as this could damage the product.

CAUTIONS

- Read instructions prior to use.
- RX Only: Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
- This device is non-sterile.
- This device is reusable.
- This device should be used by clinicians with adequate training in the use of the device.

POTENTIAL COMPLICATIONS

NONE KNOWN

INSTRUCTIONS FOR USE

- Place the Rad Trac on the procedure table underneath the mattress.

NOTE: Be sure to correctly place the Rad Trac referencing the RIGHT and LEFT stickers. The RIGHT and LEFT stickers refer to the patient’s RIGHT and LEFT as they lay on the table. It is important to have the Rad Trac positioned correctly before placing the Rad Board (Figure 2).

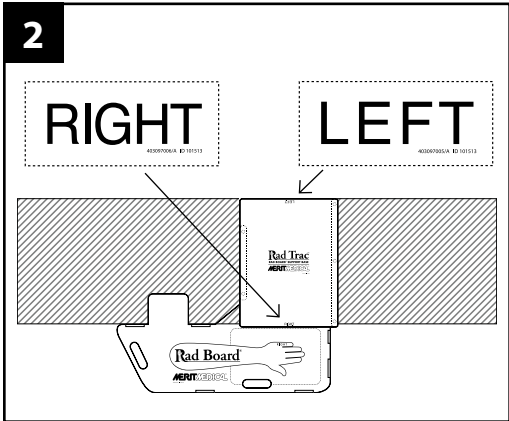
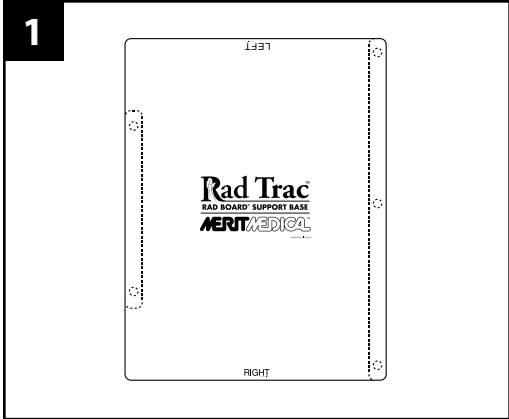
- Slide the stabilization arm of the Rad Board into the opening of the Rad Trac until the Rad Board will go no further. Be sure that the short stabilization arm towards the patient head is placed under the mattress as well. The Rad Board is now ready for use and can be easily removed from under the patient after the procedure is finished.
- The Rad Trac may be kept on the procedure table or removed after the patient is off the procedure table.
- After use, remove the Rad Trac and wipe down with a hospital approved antiseptic wipe or spray.
- Completely dry the Rad Trac and store upright or flat away from extreme heat or cold.

Manufacturer:
Merit Medical Systems, Inc., 1600 West Merit Parkway,
South Jordan, Utah, 84095 U.S.A 1-801-253-1600
U.S.A Customer service 1-800-356-3748

Authorized Representative:
Merit Medical Ireland, Ltd, Parkmore Business Park West,
Galway, Ireland

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