# VACLOK™

## INSTRUCTIONS FOR USE:

**DESCRIPTION:** A vacuum pressure syringe.

**INDICATIONS AND USAGE:** The VacLok Syringe is intended to be used by a cardiologist or radiologist during angiographic or radiologic procedures.

**CAUTION:** Federal (U.S.A.) Law restricts this device to use by or on the order of a physician. This is intended for single patient use only. Read instructions prior to use.

**INSTRUCTIONS FOR USE:** Inspect the device prior to use to verify that no damage has occurred during shipping.

1. To setup the syringe to lock, move the plunger completely forward or completely back and turn the plunger so that the locking fins can engage the stop pin.



2. To create and maintain vacuum, withdraw the syringe plunger to the desired position and rotate the plunger clockwise to position one of the locking fins behind the stop pin.

3. Turn the plunger counterclockwise to release.

4. To aspirate fluids without locking the syringe, move the plunger completely forward or completely back, turn the plunger so that the locking fins will not interfere with the stop pin.

Device is sterilized as stated on the package label.

Accuracy = ± 4%

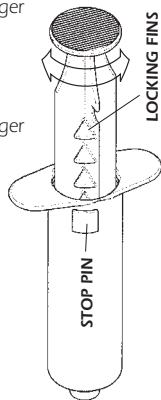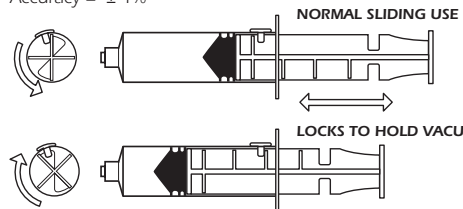**MERITMEDICAL** www.merit.com

Manufacturer: Merit Medical Systems, Inc. South Jordan, Utah 84095  
U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748

Authorized Representative:  
Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland

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