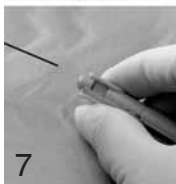
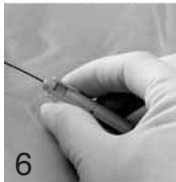
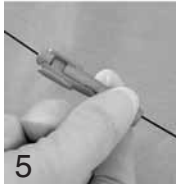
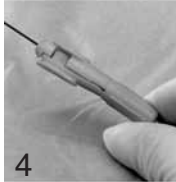
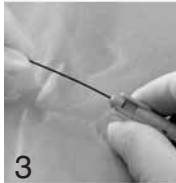
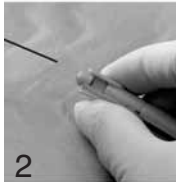
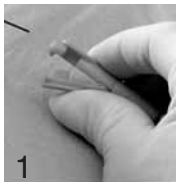
INSTRUCTIONS FOR USE

Product Description:

A guide wire torque device designed with a cylindrical shape on one end of the body and a V-shape on the other end. The elongated device body has 3 longitudinal channels that make up a single lumen for receiving and manipulating a guide wire.

Intended Use:

The Merit torque device is recommended to facilitate guide wire manipulation during interventional procedures.

Contraindications:

There are no known contraindications for these devices.

Cautions:

R_X Only.
The device is intended for single patient use only. Use once and destroy.

Read instructions for use prior to using this device. Prior to use, the device should be examined to verify functionality and ensure that its size and shape are suitable for the specific guide wire to be used.

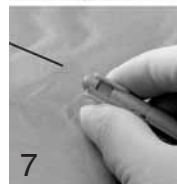
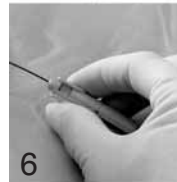
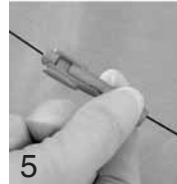
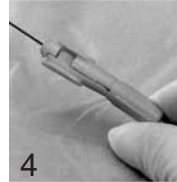
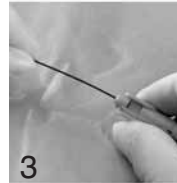
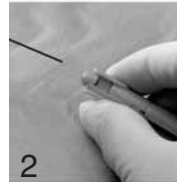
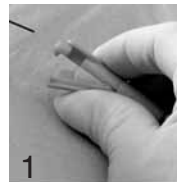
Potential Complications:

None known for the Merit H2O TORQ device. Refer to the guide wire labeling for potential complications associated with the use of their device.

Instructions for use:

1. Squeeze V-shaped finger pads of device together to align lumens.
2. Once lumens are aligned, insert proximal tip of guide wire through device.
3. Continue squeezing finger pads and slide device along guide wire until desired location is reached.
4. Release squeeze to activate locking mechanism.
5. Grasp and rotate device using ridges on back of the device to steer guide wire tip to desired position.
6. To move torque device to a new position, squeeze finger pads and slide device along the guide wire to desired position, then release.
7. To remove torque device from guide wire, squeeze finger pads and slide device off guide wire.

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