DESCRIPTION: Revolution catheter securement device.

INDICATIONS FOR USE: The Revolution catheter securement device is designed to secure a non-vascular catheter placed in a patient to help minimize catheter movement and accidental removal.

CONTRAINDICATIONS: The Revolution catheter securement device is not intended to be used on vascular catheters.

- CAUTIONS:**
- Read instructions prior to use.
 - Rx Only - Caution:** Federal (USA) law restricts this device to sale by or on the order of a physician.
 - Contents of unopened, undamaged package are sterile.


REUSE PRECAUTION STATEMENT
For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or reesterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or reesterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

INSTRUCTIONS FOR USE: Inspect the Revolution catheter securement device prior to use to verify that no damage has occurred during shipping.

A. Remove the shower lid and Revolution catheter securement device from the packaging. Use caution to ensure that you do not pull the finger rings prematurely and activate strings.

B. Before placing the Revolution catheter securement device, ensure that the skin where the device will be placed is clean and dry.

C. Position Revolution catheter securement device & secure catheter.

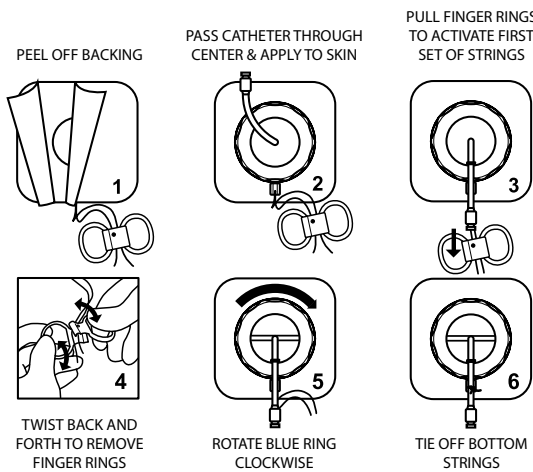D. If desired, place the supplied lid over the Revolution catheter securement device to cover the catheter and puncture site while showering.

The Revolution catheter securement device is designed to secure catheters from 6F - 24F and to remain secure up to 7 days.

CAUTIONS: In order to secure the catheter, verify that the blue ring is rotated sufficiently. Under-tightening may allow the catheter to move. Over-tightening may compress the catheter and restrict or stop fluid flow.

CAUTIONS: Verify that the catheter is not kinked after deployment.

CAUTIONS: The Revolution catheter securement device can get wet and can be worn while showering. Merit Medical recommends having the device lid in place when showering. Avoid soaking the device for long periods of time.

CAUTIONS: Avoid using alcohol or acetone based solutions when cleaning around or in the center ring of the Revolution catheter securement device. These substances may weaken the device's adhesive.

To remove the Revolution catheter securement device, remove the lid if applicable, cut the strings holding the catheter, and carefully peel the Revolution catheter securement device from the patient's skin.

Manufacturer: Merit Medical Systems, Inc. 1600 West Merit Parkway, South Jordan, Utah 84095 U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748

Authorized Representative: Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland

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Autorisierte Vertretung: Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Irland
Representante autorizado: Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Irlanda