**REVOLUTION™**

**Catheter Securement Device**

## PATIENT PAMPHLET

**MERIT MEDICAL™**

www.merit.com

CE 0086

### Revolution™ Catheter Securement Device

Your drainage catheter has been secured with a Merit Medical Revolution Catheter Securement Device, eliminating the need to suture the drainage catheter directly to your skin. The Revolution Catheter Securement Device will keep your insertion site visible for cleaning and monitoring.

The Revolution Catheter Securement Device is designed to help minimize catheter movement and accidental removal. It is made of a non-woven adhesive material that firmly secures your catheter in place for up to seven (7) days. After 7 days (sooner if the fastener becomes loose for any reason) the Revolution Catheter Securement Device will need to be replaced.

The Revolution Catheter Securement Device has a shower lid that can be placed over the device to protect your catheter when showering. Avoid soaking the Revolution Catheter Securement Device for long periods of time.

**CAUTION:** Avoid using alcohol or acetone-based solutions when cleaning around or in the center of the Revolution Catheter Securement Device. These substances may weaken the adhesive.

Inspect the Revolution Catheter Securement Device daily to make sure that:

- the catheter is secure
- the adhesive is adhering well to the skin
- the device has not become loose

Consult your healthcare provider for specific details on the daily care and maintenance of your catheter and how to inspect your catheter and exit site for possible problems.

Contact your healthcare provider if you notice any of the following:

- excess leakage
- soreness around the catheter's exit site
- movement of the catheter

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Manufacturer: Merit Medical Systems, Inc.
84095 U.S.A.
U.S.A. Customer Service 1-800-356-3748
Authorized Representative: Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland