

## INSTRUCTIONS FOR USE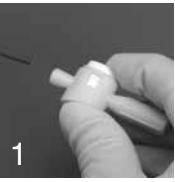

### Product Description:

A guide wire torque device designed with a cylindrical shape and a white cap grip mechanism that, when depressed, opens the lumen to allow passage of a guide wire. When the cap is released, the guide wire is gripped tightly to allow easy manipulation.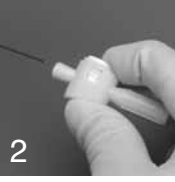

### Intended Use:

The Merit torque device is recommended to facilitate guide wire manipulation during interventional procedures.

### Contraindications:


There are no known contraindications for these devices.

### Cautions:

R<sub>x</sub> Only.  
The device is intended for single patient use only. Use once and destroy.

Read instructions for use prior to using this device. Prior to use, the device should be examined to verify functionality and ensure that its size and shape are suitable for the specific guide wire to be used.
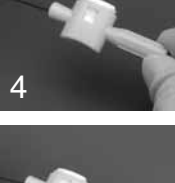


### Potential Complications:

Refer to the guide wire labeling for potential complications associated with the use of their device.

### Instructions for use:

1. Squeeze white cap of the device toward the body to align lumens.
2. Once lumens are aligned, insert proximal tip of guide wire through device.
3. Continue squeezing white cap and slide device along guide wire until desired location is reached.
4. Release squeeze to activate locking mechanism.
5. Grasp and rotate device using ridges on back of the device to steer guide wire tip to desired position.
6. To move torque device to a new position, squeeze white cap and slide device along the guide wire to desired position, then release.
7. To remove torque device from guide wire, squeeze white cap and slide device off guide wire.

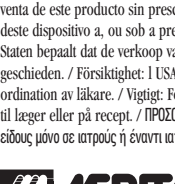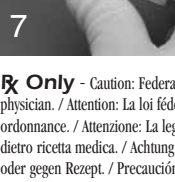**R<sub>x</sub> Only** - Caution: Federal (USA) law restricts this device to sale by or on the order of a physician. / Attention: La loi fédérale américaine limite la vente de cet article à des médecins ou sur ordonnance. / Attenzione: La legge federale degli Stati Uniti limita la vendita di quest'articolo ai medici o dietro ricetta medica. / Achtung: Das Bundesgesetz der USA limitiert den Verkauf dieses Artikels an Ärzte oder gegen Rezept. / Precaución: La ley Federal de los Estados Unidos de América (EEUU) prohíbe la venta de este producto sin prescripción médica. / Atenção: A Lei Federal dos EUA restringe a venda deste dispositivo a, ou sob a prescrição de um médico. / Waarschuwing: de wetgeving van de Verenigde Staten bepaalt dat de verkoop van dit apparaat uitsluitend door of op recept van een arts kan geschieden. / Forsikrighet: I USA forbyder federala lagar att denna utrustning säljs av läkare eller på ordination av läkare. / Vigtigt: Federale myndigheder i USA har bestemt, at denne artikel kun må sælges til læger eller på recept. / Προσοχή: Αμερικανικός ομοσπονδιακός νόμος επιτρέπει την πώληση αυτού του είδους μόνο σε ιατρούς ή έναντι ιατρικής συνταγής.

Manufacturer: Merit Medical Systems, Inc. South Jordan, Utah 84095  
U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748

Authorized Representative:  
Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland

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