BLUE DIAMOND™

INFLATION DEVICE AND FLUID DISPENSING SYRINGE

INSTRUCTIONS FOR USE

DESCRIPTION: The Blue Diamond™ Inflation Device and Fluid Dispensing Syringe by Merit Medical is a 20 ml disposable device with an integral pressure transducer, microcomputer, back-lit LCD, threaded plunger assembly with lock/release bar, a flexible high pressure extension tube, and a pressure medium pressure stopcock.

The Blue Diamond is designed to generate and monitor pressures over a range of -0.4 to +4.0 ATM/BAR (-6 to 441 PSI). The Blue Diamond syringe dispenses 0.45ml of fluid ± 0.05 ml for each 360° turn of the syringe plunger handle.

RISK ONLY CAUTION: Federal (U.S.A.) law restricts this device to use by or on the order of a physician.

INSTRUCTIONS FOR USE:

Before use, inspect the device and packaging to verify that no damage has occurred as a result of shipping.

Ensure the line is open to atmospheric pressure by opening the stopcock. Press the blue button behind the LCD to move the tubing from the inflation mode. The LCD will display "2Er0 for seconds" and then the device will be ready to use. At this point the syringe will begin its incremental time keeping.

The syringe will be set in the ATM/BAR mode when initially turned on. To change the pressure display to PSI, press and hold the blue button until "ATM/BAR" flashes four times. The user is now in "PSI" mode.

To change back to ATM/BAR, press and hold the blue button once again.

FLUID DISPENSING INDICATIONS AND USAGE:

This device is intended for use by healthcare professionals to dispense fluids to the body from the Blue Diamond syringe and monitor the pressure of that fluid. When in PSI mode, the tick marks on the left of the display that represent pressure will be limited to 300 PSI (20.4 ATM). If the Blue Diamond is pressurized past 300 PSI, the grouping of tick marks on the left will flash. The numerical digits in the center of the display will continue to show actual pressure throughout the device's pressure range (-6 to 441 PSI).

After an inflation or pressure injection has been made, a gap bar or tick mark will remain to mark the highest point of pressure. Pressing the blue button quickly will display last inflation information and a "Er" indicator in the time window. After the next inflation has been started, the last inflation tick mark will disappear.

CAUTION: If "Er" is displayed in the pressure area and a number appears in the time window, the syringe is defective. Please return the syringe to Merit Medical for credit.

NOTE: To conserve power the backlight will automatically turn off after ten minutes of inactivity. However, the microprocessor will continue to monitor the pressure. Pressing the blue button or inflating the balloon will reactivate the backlight. The device will power itself off after 90 consecutive minutes at zero pressure.

DEVICE PREPARATION:

To prep syringes simply aspirate up to 20 ml of contrast solution or other fluid into the syringe by squeezing the trigger and pulling back on the handle.

PRIMING WITH PRIMELOK™

The PrimeLok allows purging of air and debris from within the sheath of the trigger. The Blue Diamond is packaged with the PrimeLok in the disengaged position.

1. To engage PrimeLok, squeeze trigger and slide PrimeLok into slot.

2. To prep syringe, simply aspirate up to 20ml of contrast solution or fluid to be dispensed into the inflation syringe by pulling back on the plunger handle.

CAUTION: Inspect the syringe tubing and stopcock (if used) to insure that there is no air in the system.

3. Push handle against table to remove air in syringe.

4. To disengage PrimeLok, squeeze trigger and slide PrimeLok out of slot. This will allow the plunger to lock in position and the device is now ready for use.

NOTE: The PrimeLok must be disengaged before pressure can be maintained by the lock/release mechanism.

RECOMMENDED SEPARATION DISTANCES BETWEEN PORTABLE AND MOBILE RADIO FREQUENCY (RF) COMMUNICATIONS EQUIPMENT AND THE BLUE DIAMOND™ INFLATION DEVICE

The Blue Diamond Inflation Device is intended for use in an electromagnetic environment in which RF-induced disturbances are controlled. The user of the Blue Diamond Inflation Device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Blue Diamond Inflation Device as recommended below, according to the maximum output power of the communications equipment.

Separation distance according to frequency of transmitter (in meters) m

Rated maximum output power of transmitter (in watts)

W

150 kHz to 80 MHz

80 MHz to 800 MHz

800 MHz to 2.5 GHz

d = [1,2] √P

d = [1,2] √P

d = [2,3] √P

0.01

0.12

0.12

0.2

0.01

0.37

0.37

0.7

0.1

1.2

1.2

2.3

10

3.7

3.7

7.3

100

12

12

23

For transmitters rated at a maximum output power not listed above, the recommended separation distance (d) in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 - At 80 MHz and 800 MHz, the separation distance for the higher frequency applies.

NOTE 2 - These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

GUIDANCE AND MANUFACTURER'S DECLARATION - ELECTROMAGNETIC IMMUNITY

The EQUIPMENT is intended for use in the electromagnetic environment specified below.

The customer or the user of the EQUIPMENT should assure that it is used in such an environment.

Immunity Test

IEC 60061 test level

Compliance level

Electromagnetic environment-guidance

Electrostatic discharge (ESD)

±6 kV contact

±8 kV air

Increasing relative humidity will reduce the potential for ESD related difficulties

Users should follow local guidelines and practices regulating the disposal of infected waste products.

STERILE | EO EtO Sterilized

()

Interference may occur in the vicinity of equipment marked with this symbol

Caution! Consult accompanying documents

Non-pyrogenic

Sterile if package is unopened and undamaged.

U.S. Patent Nos. 5,047,015, 5,057,078, 5,135,488 Other U.S. and Foreign Patents Pending.

CE 0086

Manufacturer: Merit Medical Systems, Inc., 1600 W. Merit Pkwy, So. Jordan, UT 84095 U.S.A. 1-801-253-1600

Authorized Representative: Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland

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