REPLACEABLE PRESSURE INFUSOR BAG

INSTRUCTIONS FOR USE

DESCRIPTION OF THE PRODUCT

The pressure infusor bag (PIB) consists of a bag with a bladder, an envelope for holding the fluid bag, a pressure gauge, and tubing with an inflation bulb and a stopcock. Once the fluid bag is loaded in the envelope, the bag will then be inflated to the desired pressure.

INTENDED USE

A pressure infusor bag is used to apply pressure to a sealed bag of sterile fluid assisting in the infusion of the fluid.

CONTRAINDICATIONS

There are no Contraindications or Warnings identified for this device.

CAUTIONS

- Read instructions prior to use.
- Ensure adequate inflation pressure during use.
- **Rx Only** – Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

INSTRUCTIONS FOR USE

1. Insert intravenous bag between mesh and pressure bag.
2. Slide hanger loop through top of intravenous bag.
3. Inflate: Close valve on inflation bulb. Pump inflation bulb until gauge indicates the proper pressure.
4. Rotate stopcock to "Maintain Pressure" position.
5. Deflate: Open valve on inflation bulb.

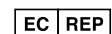
Storage Conditions: Room temperature, ventilation, dry, away from light.

Transportation Conditions: Do not expose to excessive heat or humidity during transportation.

www.merit.com

Merit Medical Systems, Inc.
1600 West Merit Parkway, South Jordan, UT 84095 U.S.A.
1-801-253-1600 - U.S.A. Customer Service 1-800-356-3748

Merit Medical Ireland Ltd,
Parkmore Business Park West, Galway, Ireland

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Merit Medical Ireland Ltd,
Parkmore Business Park West, Galway, Irlande

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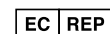
Merit Medical Ireland Ltd,
Parkmore Business Park West, Galway, Irlanda

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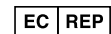
Merit Medical Ireland Ltd,
Parkmore Business Park West, Galway, Irland

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