DRAINAGE CATHETER

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

The One-Step is a short term drainage catheter with introducer needle and vent plug.

INTENDED USE

The One-Step Centesis Drainage Catheter is intended for the percutaneous drainage of fluids.

CAUTIONS

- Read instructions prior to use
- Rx Only Caution:** Federal Law (USA) restricts this device to sale by or on the order of a physician.
- Contents of unopened, undamaged package are sterile using ethylene oxide.
- Do not leave in patient longer than 24 hours.
- Store in a cool dry place
- Only persons knowledgeable of the risks involved and qualified in the procedures performed should use this device.

REUSE PRECAUTION STATEMENT

For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or crossinfection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

INSTRUCTIONS FOR USE

Inspect the device prior to use to verify that no damage has occurred during shipping.

- Remove catheter and needle from protective boot.
- Insert catheter into fluid collection site using standard insertion technique.
- When catheter is in appropriate position remove needle from catheter.
- Attach device with standard luer fitting to catheter hub to remove fluid.
- Following the procedure gently withdraw the catheter and dress the site accordingly.

Manufacturer: www.merit.com
Merit Medical Systems, Inc., 1600 West Merit Parkway
South Jordan, Utah 84095 U.S.A. 1-801-253-1600
U.S.A. Customer Service 1-800-356-3748

EC REP Authorized Representative:
Merit Medical Ireland Ltd, Parkmore Business Park West,
Galway, Ireland

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