

### INSTRUCTIONS FOR USE

#### PRODUCT DESCRIPTION

The Merit Coronary Control Syringe contains a calibrated hollow barrel into which is inserted a closely fitted movable plunger and tip or seal. The barrel contains a fixed male luer connector or male luer rotating adaptor, which are compatible with female luer hubs.

#### INDICATIONS FOR USE

The Merit Coronary Control Syringe is intended to be used for the intra-arterial or intravenous administration of radiographic contrast media. The syringe can also be used to inject fluids into, or withdraw fluids from, the body.

#### CONTRAINDICATIONS

None known

#### PRECAUTIONS

Carefully read instructions before using product. If product is being used in conjunction with other manufacturers' components, also read their Instructions for Use.

Use Proper aseptic techniques while handling product.

Inspect device prior to use to verify that no damage has occurred during shipping.

**Rx ONLY Caution:** Federal (USA) law restricts this device to sale by or on the order of a physician trained and/or experienced in the use of the device.

#### REUSE PRECAUTIONS STATEMENT

For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

#### INSTRUCTIONS FOR USE

1. Using aseptic technique, open the package containing the sterile product.
2. Inspect the device prior to use to verify that no damage has occurred during shipping.
3. To prep the syringe, using one or two hands, pump the piston back and forth several times to ensure that the piston moves smoothly.
4. To fill the syringe, pull plunger back.
5. Connect the syringe head to the female luer by turning the syringe clockwise. DO NOT OVER-TIGHTEN. Do not use any instrument to tighten connections.
6. To inject, push the syringe plunger, injecting the appropriate amount of fluid.

**Note:** Shelf life: 3 years.

**Storage Conditions:** Store in a cool, dry place.

**Transportation Conditions:** Do not expose to excessive heat or humidity during transportation.



Single Use Only / Do Not Reuse



Contents are non-pyrogenic



Do Not Re-Sterilize



Do not use if package is damaged



www.merit.com



Manufacturer:  
Merit Medical Systems, Inc.  
1600 West Merit Parkway  
South Jordan, Utah 84095 U.S.A.  
1-801-253-1600  
U.S.A Customer Service 1-800-356-3748



Authorized Representative:  
Merit Medical Ireland Ltd, Parkmore Business Park West  
Galway, Ireland  
EC Customer Service +31 43 3588222