



LITTLE THINGS MAKE A BIG DIFFERENCE


 **Sea Dragon 2**[™] Torque Device

Manipulate the guide wire more easily and quickly during your interventional procedures



SIMPLIFY TORQUE DEVICE MANIPULATION

Operable with one hand, this ergonomic torque device is designed to enhance manipulation of the guide wire tip. To position, reposition, or remove the device, simply squeeze the white top cap.

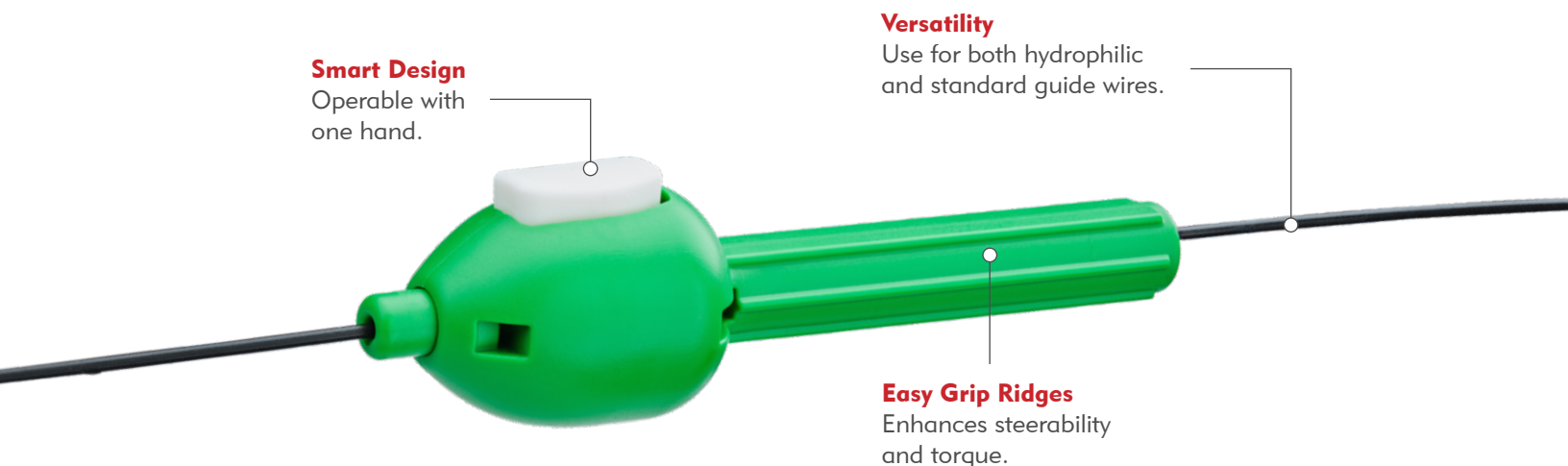

SQUEEZE:
Squeeze the white top cap toward the green body to align lumens



LOAD:
Insert proximal guide wire tip through the device



RELEASE:
Release white top cap to activate locking mechanism



TORQUE:
Easy grip ridges enhances steering and torque performance

Catalog Number	Description	Accommodates guide wires
TD500	Guide wire torque device	0.014" (0.36 mm) to 0.038" (0.097 mm)

Packaged individually, 10 units per box.

For more information on Laureate and InQwire guide wires go to Merit.com

MERITOLAUREATE®



InQwire



Before using refer to Instructions for Use for indications, contraindications, warnings, precautions, and directions for use.



Understand. Innovate. Deliver.™

merit.com

Merit Medical Systems, Inc.
1600 West Merit Parkway
South Jordan, Utah 84095
Main: +1.801.253.1600
Customer Service: +1.801.208.4300

Merit Medical Europe, Middle East, & Africa (EMEA)
Amerikalaan 42, 6199 AE
Maastricht-Airport
The Netherlands
+31 43 358 82 22

Merit Medical Ireland Ltd.
Parkmore Business Park West
Galway, Ireland
+353 (0) 91 703 733