PATIENT GUIDE

Please consult the Aspira® Drainage System Instructions For Use for warnings, cautions, and full instructions for use.


STEP 1: Prepare the Workspace

- 1 Clear space on table.
- 2 Clean surface with disinfecting wipe.
- 3 Wash hands with warm soapy water. Scrub for one full minute. Dry with a clean towel.
- 4 Place supplies on cleaned area.

STEP 2: Drainage Procedure

NOTE: If the catheter is taped to the skin with tape strips, remove the tape.
NOTE: If the catheter is damaged, place the slide clamp (in kit) between the catheter damage and the exit site. Contact your doctor.

- 1 Peel open the pouch with the drainage bag or bottle.
- 2 Remove and discard valve cap from catheter valve.
- 3 Wipe catheter valve with alcohol pad.
- 4 Connect drainage line to catheter. You should hear or feel a click when secure.
NOTE: Steps 5 and 6 are for using an Aspira Drainage Bag. If you are using an Aspira Drainage Bottle, follow steps 7 and 8.
- 5 Place bag on a flat surface at least arms length **below chest or abdomen**.
- 6 Squeeze pump one time. Let fluid drain until bag is full or fluid stops flowing. If fluid does not flow, refer to troubleshooting section.
- 7 Place bottle upright on a flat surface with the bottle handle accessible.
- 8 Open the bottle by rotating the handle counterclockwise until the arrow in the handle lines up with the tubing (approximately half a turn). Let fluid drain until the bottle is full or stops flowing. If fluid does not flow, refer to troubleshooting section.
CAUTION: While the bottle is connected to the catheter, do not rotate the handle past where the arrow in the handle lines up with the tubing (approximately half a turn) until you are ready to empty the bottle. If the handle is rotated too far while connected to the catheter, air could enter the system causing a pneumothorax. If air enters the system, disconnect the tubing from the catheter and use another bottle.
NOTE: When rotating the handle a bump may be felt. This is an indication to stop rotating the handle until ready to remove the handle completely. Do not rotate the handle past this feature until the bottle has been disconnected from the catheter and you are ready to empty the contents of the bottle.
- 9 When fluid stops or the bag or bottle fills to 1,000 mL from the chest or from the abdomen, disconnect drainage line from catheter.
- 10 Wipe catheter valve with new alcohol pad.
- 11 Place new valve cap on catheter valve.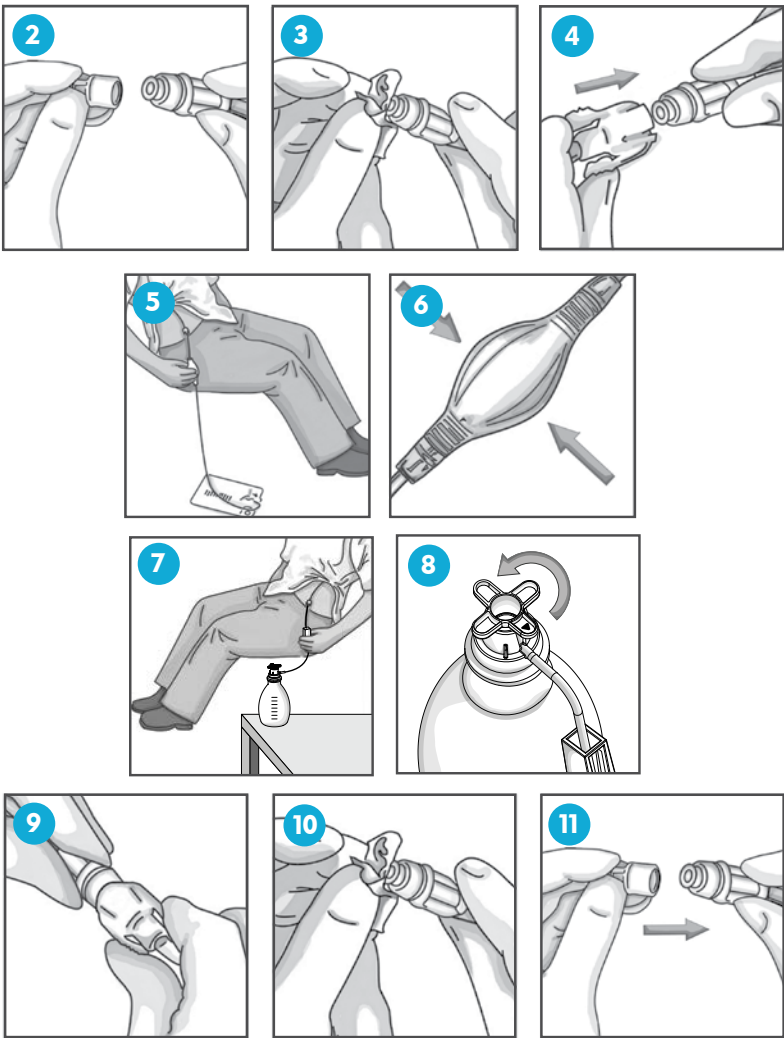


STEP 3: Dressing Procedure

If it is not time to change your dressing, tape the catheter to the skin and skip to step 4.


NOTE: Look at the exit site and skin around it. If you notice any redness, swelling, oozing, or have pain at the exit site, finish draining and call your doctor.

- 1 Peel open the pouch with the dressing supplies.
NOTE: If using an Aspira Drainage Bottle, the dressing supplies come in the same pouch as the drainage bottle.
- 2 Remove the gloves from the pouch.
- 3 Pick up glove at cuff end and place on hand.
- 4 Pick up second glove with the gloved hand and place it on your other hand.
- 5 Remove sterile sheet from the pouch and place it on prepared workspace.
- 6 Lay sheet flat, so you can see dressing supplies.
- 7 Clean skin around exit site with alcohol pad.
NOTE: Make sure exit site and skin are dry before proceeding.
- 8 Place split gauze pad on skin around catheter.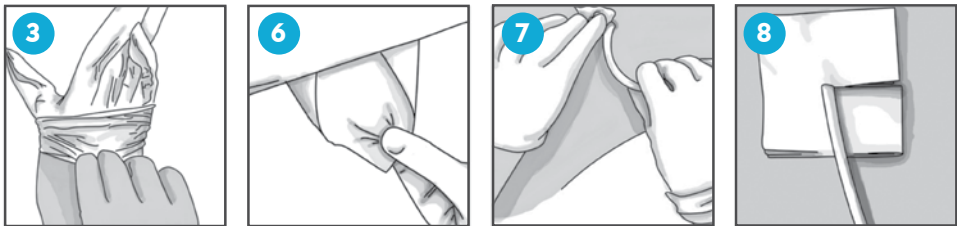


Dress At Every Drainage

- 1 Coil catheter on top of split gauze pad. Place gauze on top of the catheter.
- 2 Place clear dressing over catheter and gauze.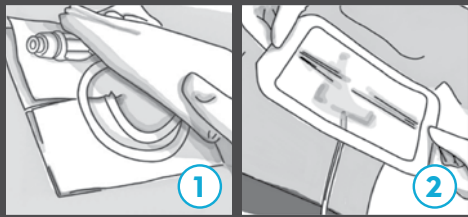


Dress Weekly

- 1 Place gauze on top of catheter. Place clear dressing over catheter and gauze.
- 2 Tape exposed catheter to skin.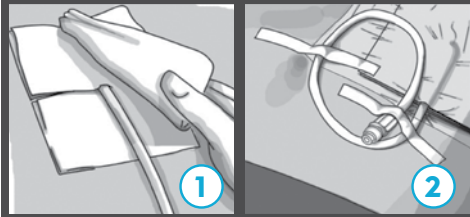


STEP 4: Discarding Fluid and Used Supplies

- 1 If using a drainage bag, tear or cut the corner of the drainage bag over the toilet. Empty fluid into toilet.
- 2 If using a drainage bottle, remove the handle of the drainage bottle over the toilet by rotating the handle counterclockwise approximately two full turns. Empty fluid into toilet.
- 3 Throw drainage bag, or bottle, and used supplies into trash.
- 4 Wash hands.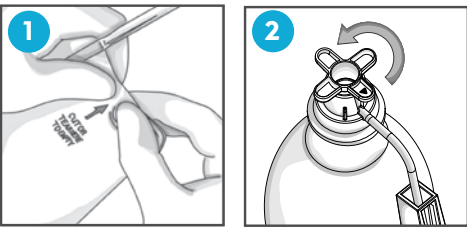

NOTE: If the catheter is damaged, place slide clamp provided in drainage kit between the damage and catheter exit site. Call your doctor.

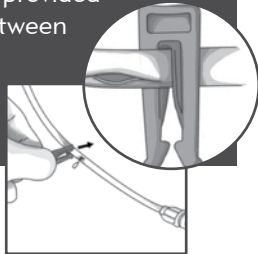CAUTION: If you have any of the following symptoms, complete the drainage and contact your doctor, visit an emergency room, or call 911:

- Redness, swelling, oozing, or have pain around the exit site.
- Shortness of breath that isn't relieved after draining 1,000 mL from the chest or 2,000 mL from the abdomen at one time.
- Ongoing chest pain or shortness of breath but little or no fluid is draining from the catheter.
- Less than 25–50 mL drain during each drainage for three drainages in a row.
- The appearance of your fluid changes a lot between drainages.
- Body temperature above 100.5°F (38°C).

CAUTION: Do not drain more than 1,000 mL from the chest or 2,000 mL from the abdomen in any one drainage.

CAUTION: If fluid spills, clean using soap and warm water for clothing and skin, and use bleach for household items.

NOTE: Do not recycle used drainage bags or bottles.

NOTE: If the catheter moves in and out of the exit site, call your doctor.

NOTE: If you experience dizziness, lightheadedness, or blurred vision, you may have low blood pressure. Stop draining. Call your doctor.

NOTE: Always follow the recommended cautions and orders of your healthcare provider.

TROUBLESHOOTING: If the flow of fluid does not start, check the connection and ensure there is no kink or loop in the drainage line.
If using a drainage bag, gently squeeze the pump again. If using a drainage bottle, make sure the roller clamp is fully open on the bottle drainage line. If this still does not work, try using a new drainage kit.
If unable to start flow, contact your doctor.

For additional information, scan the QR code or visit myaspira.com.

Remove dressing only if:

- 1. It has been one week since you changed the dressing.
- OR -
- 2. Your doctor recommended changing the dressing at every drainage.
- OR -
- 3. The area under the dressing is moist.

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