



Predictable. Targeted. Established.™

Embosphere[®]

Microspheres

PREDICTABLE

Embosphere Microspheres temporarily compress up to 33% for smooth microcatheter passage. Once through the microcatheter, they return to their original spherical shape and stated diameter for predictable, reliable delivery.¹

TARGETED

Embosphere Microspheres show a direct correlation between the level of arterial occlusion and the size of the particles used, allowing for consistent and reliable targeted occlusion.^{2,3}

ESTABLISHED

Embosphere Microspheres are the most clinically studied and clinically utilised spherical embolic, having been proven:


- In over 350,000 procedures⁴
- With +20 years' clinical experience⁴
- In more than 200 clinical articles⁴


Non-Aggregating Properties

The hydrophilic surface and spherical shape of Embosphere Microspheres prevent aggregation in the catheter lumen and vasculature.

Superior Rebound Capability

Embosphere Microspheres' elastic properties allow temporary compression of up to 33% to facilitate smooth microcatheter passage (A).¹ Once through the microcatheter, Embosphere Microspheres return to their original spherical shape and stated diameter (B) for predictable, targeted delivery.¹

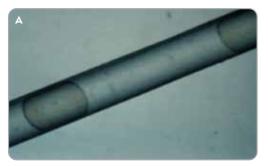

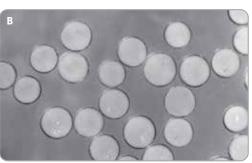







Durable Occlusion

Embosphere Microspheres are biocompatible and nonresorbable with cell adhesion properties that enable complete and durable mechanical occlusion.

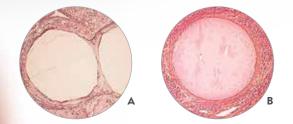

Embosphere Microspheres at six months (A) and six years (B) after embolisation of facial AVM. Images courtesy of Dr. Alex Laurent, Hôpital Lariboisière, Paris, France

Percentage of Patients with Complete Infarction Rates of Overall Uterine Fibroid Tumor Burden

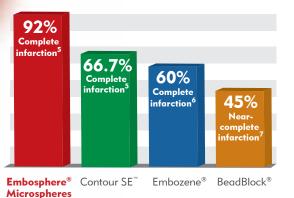

Complete infarction of all fibroids leads to better long-term clinical outcomes, including a higher rate of symptom control and a lower rate of additional gynecologic intervention compared with

incomplete infarction of fibroid tissue.⁸

REFERENCES

1. Laurent, et al. Trisacryl Gelatin Microspheres for Therapeutic Embolization, 1: Development and In Vitro Evaluation. Am J Neuroradial. 1996 Mar; 17:533-540

With Embosphere Microspheres, 92.3% of patients experienced 100% infarction of their entire uterine fibroid

tumor burden.

- Pelage JP, et al. Uterine Artery Embolization in Sheep: Comparison of Acute Effects with Polyvinyl Alcohol Particles and Calibrated Microspheres. Radiol. 2002;224:436-45
 Verret V, et al. The Arterial Distribution of Embozene and Embosphere Microspheres in Sheep Kidney and Uterus Embolization Models. J Vasc Interv Radiol. 2011 Feb;22(2):220-8
- Verret V, et al. The Arterial Distribution of Embozene and Embosphere Microspheres in Sheep Kidney and Uterus Embolization Models. J Vasc Interv Radiol. 2011 Feb;22(2):220-{
 Data on file
- 5. Siskin G, et al. Leiomyoma Infarction after Uterine Artery Embolization: A Prospective Randomized Study Comparing Tris-acryl Gelatin Microspheres versus Polyvinyl Alcohol Microspheres. J Vasc Interv Radiol. 2008; 19:42-46
- 6. Smeets AJ, et al. Embolization of Uterine Leiomyomas with Polyzene F-coated Hydrogel Microspheres: Initial Experience. J Vasc Interv Radiol. 2010 Dec;21(12):1830-4
- 7. Chrisman HB, et al., Prospective Evaluation of the Embolic Agent Bead Block in the Treatment of Uterine Leiomyomas with Uterine Artery Embolization. J Vasc Interv Radiol. 2010 Apr;21(4):484-9
- 8. Katsumori T, et al. Infarction of Uterine Fibroids After Embolization: Relationship Between Postprocedural Enhanced MRI Findings and Long-Term Clinical Outcomes. Cardiovasc Intervent Radiol. 2008;31:66

Embosphere®

Microspheres

ORDERING INFORMATION

	Syringe Size Range (µm)	Colour Code	1 mL Syringe Product Number	2 mL Syringe Product Number
S	50-100	Gray		S020GH
Ú	40-120	🛑 Orange		S120GH
כ	100-300	Yellow	S210GH	S220GH
Ζ	300-500	🔵 Blue	S410GH	S420GH
Y	500-700	🛑 Red	S610GH	S620GH
∽	700-900	Green	S810GH	S820GH
^	900-1200	Purple	S1010GH	S1020GH

Embosphere Microspheres supplied in sterile saline in 20 mL syringe. Sold 1 per box.



SPECIFIC SIZE RANGES

Embosphere Microspheres are available in seven specific size ranges to achieve predictable and efficient vessel occlusion.

USAGE INSTRUCTIONS

Suspension

- 1. Inject an equivalent amount of CM to the NaCl provided in the syringe (50/50).
- **2**. Spheres will begin to float after the injection.
- Gently invert the syringe, after 3 minutes you will get a durable suspension.

Biosphere Medical, S.A. Parc des Nations - Paris Nord 2 383 rue de la Belle Etoile 95700 Roissy en France France



Indications For Use

Embosphere Microspheres are designed to occlude blood vessels, for therapeutic or preoperative purposes, in the following procedures:

- Embolisation of hypervascular tumours and processes, including uterine fibroids, meningiomas, liver tumours.
- Embolisation of the prostate arteries for relief of symptoms related to Benign Prostatic Hyperplasia.
- Embolisation of arteriovenous malformations.
- Haemostatic embolisation.

The above suggestions and other information are for the practitioner's convenience and for general information purposes only. This information does not constitute medical or legal advice, nor is it meant to endorse or guarantee the suitability of any of the referenced products or methods for any specific patient or procedure. Before using any product, refer to the Instructions for Use (IFU) for indications, contraindications, warnings, precautions, and directions for use.



Understand. Innovate. Deliver.**

merit.com

Merit Medical Systems, Inc. 1600 West Merit Parkway South Jordan, Utah 84095 1.801.253.1600 1.800.35.MERIT Merit Medical Europe, Middle East, & Africa (EMEA) Amerikalaan 42, 6199 AE Maastricht-Airport The Netherlands + 31 43 358 82 22 Merit Medical Ireland Ltd. Parkmore Business Park West Galway, Ireland + 1800 553163