



1  
Remove the plunger from a standard 30 mL syringe.



2  
Remove the screw-top cap from the vial and transfer the **Bearing nsPVA Embolization Particles** into the barrel of the syringe. Replace the plunger on the syringe.



3  
Aspirate **10 mL** of 0.9% NaCl and **10 mL** of contrast.



4  
To ensure proper hydration and suspension, gently agitate the saline/contrast mixture of **Bearing nsPVA Embolization Particles** and then wait **2-3 minutes**, prior to injection.



5  
Attach the 30 mL syringe to one port of a luer-lock 3-way stopcock. Attach a 1 mL or 3 mL injection syringe to another port on the stopcock.

**NOTE:** If desired, a catheter may be attached to the remaining port on the stopcock.



6  
**NOTE:** Purge all air from the system prior to injection.

Draw the **Bearing nsPVA Embolization Particles** saline/contrast mixture into the injection syringe slowly and gently to minimize the potential of introducing air into the system.



7  
Inject the **BEARING nsPVA Embolization Particles** saline/contrast mixture into the delivery catheter under fluoroscopic visualization using a slow pulsatile injection while observing the contrast flow rate.

**NOTE:** If there is no effect on the flow rate, repeat the delivery process with additional injections of the Bearing/saline/contrast mixture. Consider using larger sized Bearing nsPVA Embolization Particles if the initial injections do not alter the contrast flow rate. Exercise conservative judgment in determining the embolization endpoint.

**NOTE:** Upon completion of the treatment, remove the catheter while maintaining gentle suction so as not to dislodge Bearing nsPVA Embolization Particles still within the catheter lumen.



Understand. Innovate. Deliver.™

Merit Medical Systems, Inc.  
1600 West Merit Parkway  
South Jordan, Utah 84095  
1.801.208.4300  
1.800.35.MERIT

Merit Medical Europe, Middle East, & Africa (EMEA)  
Amerikalaan 42, 6199 AE Maastricht-Airport  
The Netherlands  
+31 43 358 82 22

Merit Medical Ireland Ltd.  
Parkmore Business Park West  
Galway, Ireland  
+353 (0) 91 703 733

Merit.com

Austria  
0800 295 374

Belgium  
0800 72 906 (Dutch)  
0800 73 172 (Français)

Denmark  
80 88 00 24

Finland  
0800 770 586

France  
0800 91 60 30

Germany  
0800 182 0871

Ireland (Republic)  
1800 553 163

Italy  
800 897 005

Luxembourg  
8002 25 22

Netherlands  
0800 022 81 84

Norway  
800 11629

Portugal  
308 801 034

Russia  
+7 495 221 89 02

Spain  
+34 911238406

Sweden  
020 792 445

Switzerland  
(Deutsch)  
+41 225180252  
(Français)  
+41 225948000  
(Italiano)  
+41 225180035  
UK  
0800 973 115