


RESOLVE ConvertX[®]

Nephroureteral Stent System

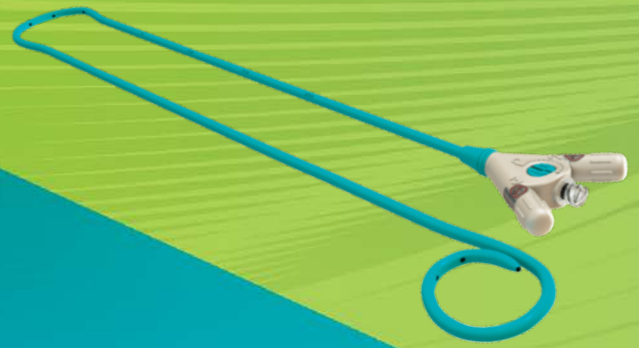


The only product that
eliminates a catheter-to-stent exchange procedure
in the treatment of severe ureteral obstructions

- Treat Patients with One Procedure, Not Two
- Reduce Cost, Time and Risk¹
- Intuitive Stent Conversion



STENT RELEASE
in Less than One Minute²

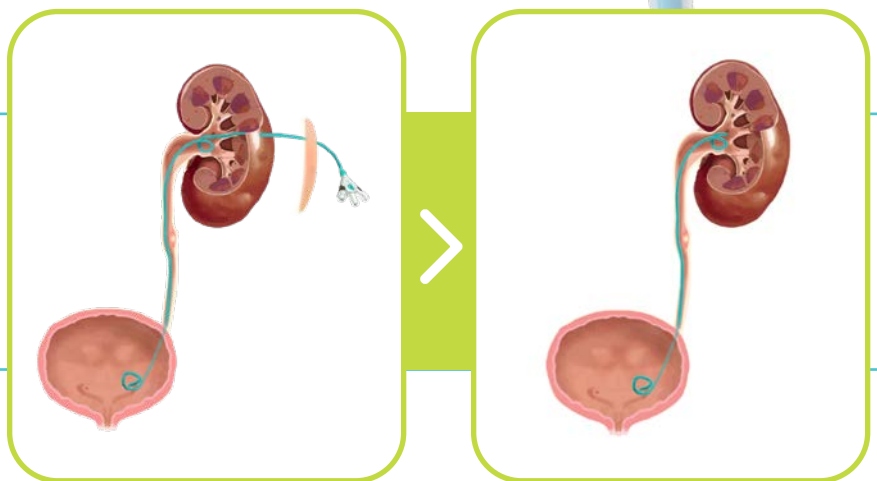

¹ Due to eliminating an interventional exchange procedure.
² Data on file.

Powered by



Simple Conversion in Office or at Bedside



1 Rotate stopcock to unlocked position



2 Unscrew cap and remove loop suture



3 Unscrew cap, pull release wire, and remove external drainage catheter



REFERENCE NUMBER	STENT LENGTH	DISTAL STENT DIAMETER	PROXIMAL STENT DIAMETER	RELEASABLE DRAINAGE CATHETER LENGTH
RCX10-8-038	10 cm	8.3 F	10.3 F	20 cm
RCX12-8-038	12 cm	8.3 F	10.3 F	20 cm
RCX14-8-038	14 cm	8.3 F	10.3 F	20 cm
RCX20-8-038	20 cm	8.3 F	10.3 F	30 cm
RCX22-8-038	22 cm	8.3 F	10.3 F	30 cm
RCX24-8-038	24 cm	8.3 F	10.3 F	30 cm
RCX26-8-038	26 cm	8.3 F	10.3 F	30 cm
RCX28-8-038	28 cm	8.3 F	10.3 F	30 cm

REFERENCE NUMBER	STENT LENGTH	DISTAL STENT DIAMETER	PROXIMAL STENT DIAMETER	RELEASABLE DRAINAGE CATHETER LENGTH
RCX10-10-038	10 cm	10.3 F	10.3 F	20 cm
RCX12-10-038	12 cm	10.3 F	10.3 F	20 cm
RCX14-10-038	14 cm	10.3 F	10.3 F	20 cm
RCX20-10-038	20 cm	10.3 F	10.3 F	30 cm
RCX22-10-038	22 cm	10.3 F	10.3 F	30 cm
RCX24-10-038	24 cm	10.3 F	10.3 F	30 cm
RCX26-10-038	26 cm	10.3 F	10.3 F	30 cm
RCX28-10-038	28 cm	10.3 F	10.3 F	30 cm

All stents are compatible with standard 0.038" (0.97 mm) guidewires.

Before using refer to Instructions for Use for indications, contraindications, warnings, precautions, and directions for use.

Powered by



merit.com

Merit Medical Systems, Inc.
1600 West Merit Parkway
South Jordan, Utah 84095
Main: +1.801.253.1600
Customer Service: +1.801.208.4300

Merit Medical Europe, Middle East, & Africa (EMEA)
Amerikalaan 42, 6199 AE
Maastricht-Airport
The Netherlands
+31 43 358 82 22

Merit Medical Ireland Ltd.
Parkmore Business Park West
Galway, Ireland
+353 (0) 91 703 733