


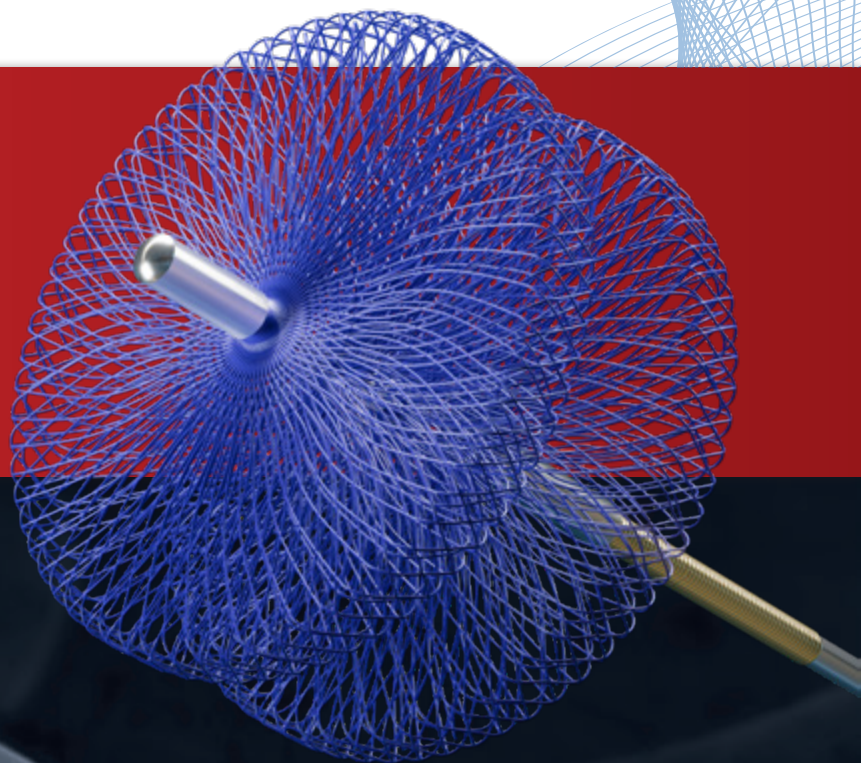

SIEGE™

VASCULAR PLUG

A faint, light blue illustration of a blood vessel system, showing a main vessel branching into several smaller ones. A small, dark, cylindrical object, representing the vascular plug, is shown inserted into one of the smaller branches.

**CONFIDENCE
DELIVERED**


TREAT WITH CONFIDENCE

The Siege Vascular Plug is a self-expanding vascular implant designed for peripheral arterial embolization in vessels measuring 1.5–6.0 mm in diameter. Multiple features support smooth advancement through the vasculature, offering complete and rapid occlusion where you need it.¹

PRODUCT BENEFITS¹

Flexibility

Conformable nitinol braid maintains apposition within the artery in both straight and curved anatomy.

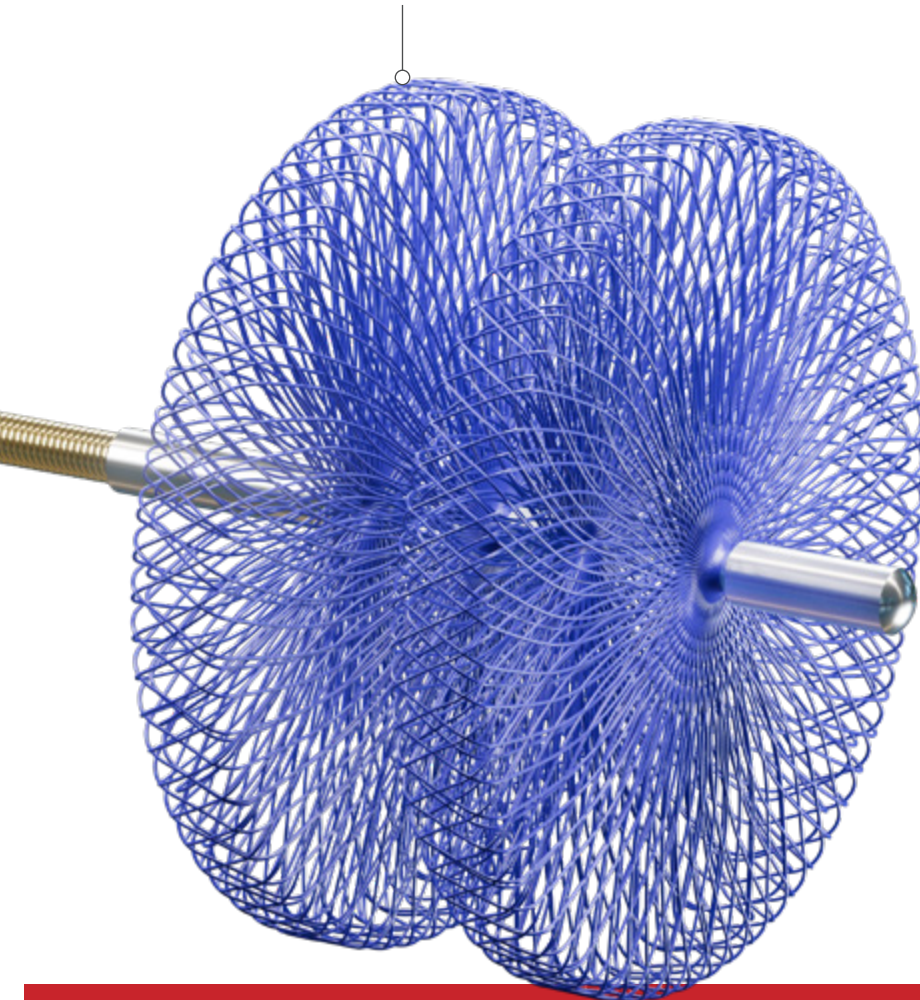


Enhanced Deliverability

Hydrophilic coating for smooth advancement through tortuous anatomy.



Precise Placement

Recapture, reposition, and re-deploy where needed in all three sizes.*

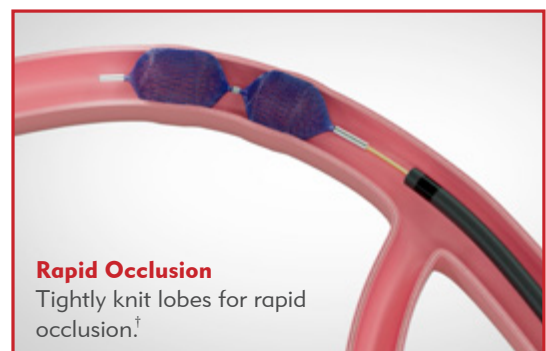

Rapid Occlusion

Tightly knit lobes for rapid occlusion.[†]

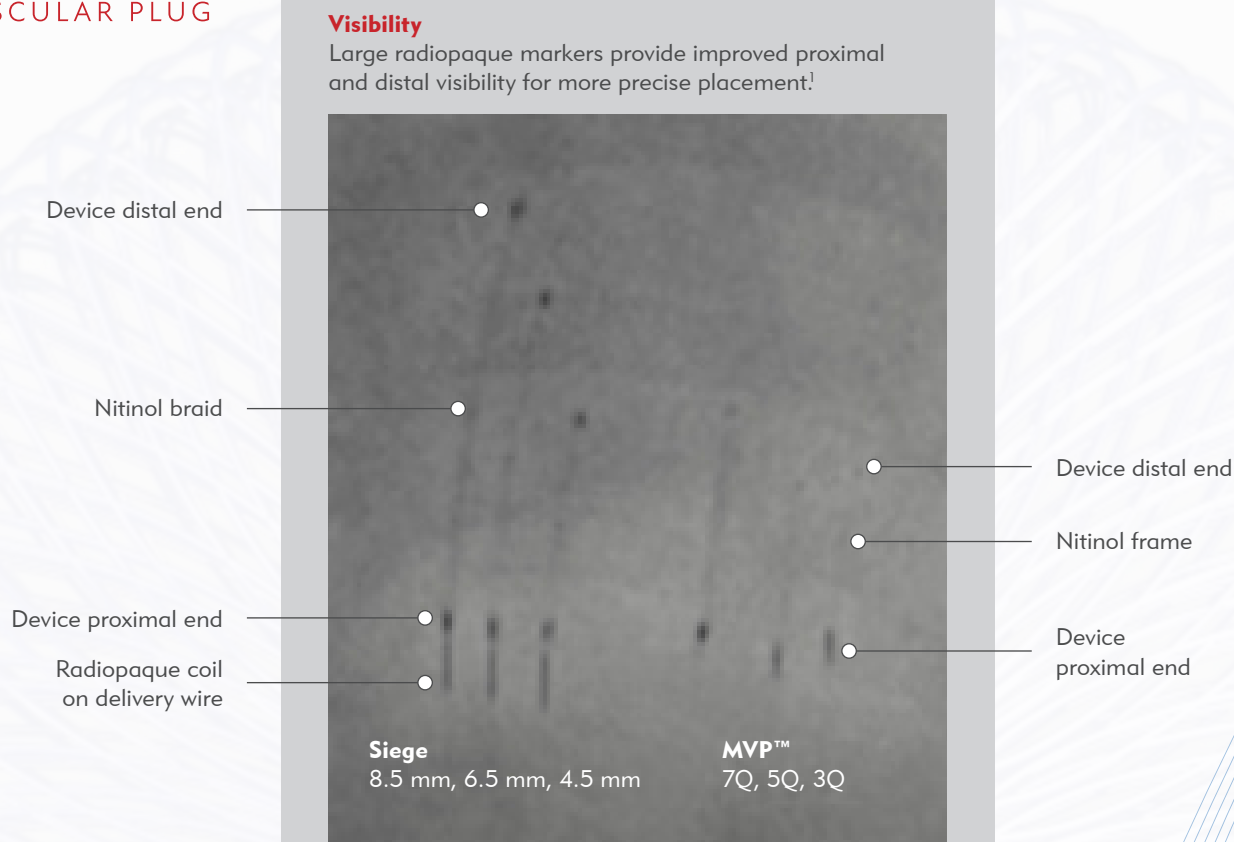


Figure 1. Still from fluoroscopy (7.5 frames/s)

COMPATIBILITY


The Siege is compatible with commonly used 0.021" and 0.027" microcatheters and can treat larger arteries than the MVP Micro Vascular Plug System.[‡]

Product Name	Siege Vascular Plug Catalog Number		Minimum Inner Diameter	Maximum Length
Boston Scientific Renegade™ STC 18 (2.4F/3.0F)	SVP2.5-0.021	SVP4-0.021	0.021" (0.533 mm)	175 cm
Medtronic Rebar™ (2.4F/2.7F)	SVP2.5-0.021	SVP4-0.021	0.021" (0.533 mm)	175 cm
Merit Medical SwiftNINJA® (2.4F/2.6–2.9F)	SVP2.5-0.021	SVP4-0.021	0.021" (0.533 mm)	175 cm
Stryker Trevo® Pro 18 (2.4F/2.7F)	SVP2.5-0.021	SVP4-0.021	0.021" (0.533 mm)	175 cm
Terumo PROGREAT® (2.4F/2.9F)	SVP2.5-0.021	SVP4-0.021	0.021" (0.533 mm)	175 cm
Boston Scientific Renegade™ HI-FLO™ (2.8F/3.0F)	SVP6-0.027		0.027" (0.686 mm)	175 cm
Medtronic Rebar™ (2.8F/2.8F)	SVP6-0.027		0.027" (0.686 mm)	175 cm
Stryker Excelsior® XT-27® (2.7F/2.9F)	SVP6-0.027		0.027" (0.686 mm)	175 cm
Terumo PROGREAT® (2.8F/3.0F)	SVP6-0.027		0.027" (0.686 mm)	175 cm

Compatibility with the microcatheters listed has been established. Physicians should exercise their clinical judgement for the selection and use of microcatheters. Refer to the Instructions for Use for each product listed. Manufacturers may make changes to their microcatheters without notice, which may impact the suitability for use with Siege Vascular Plugs.

WARNING: Compatibility with microcatheters not listed has not been established. The use of other microcatheters may result in an inability to deliver, deploy, or recapture the device.

Merit Medical provides no warranty, express or implied, on third-party microcatheters used with Merit Medical products.

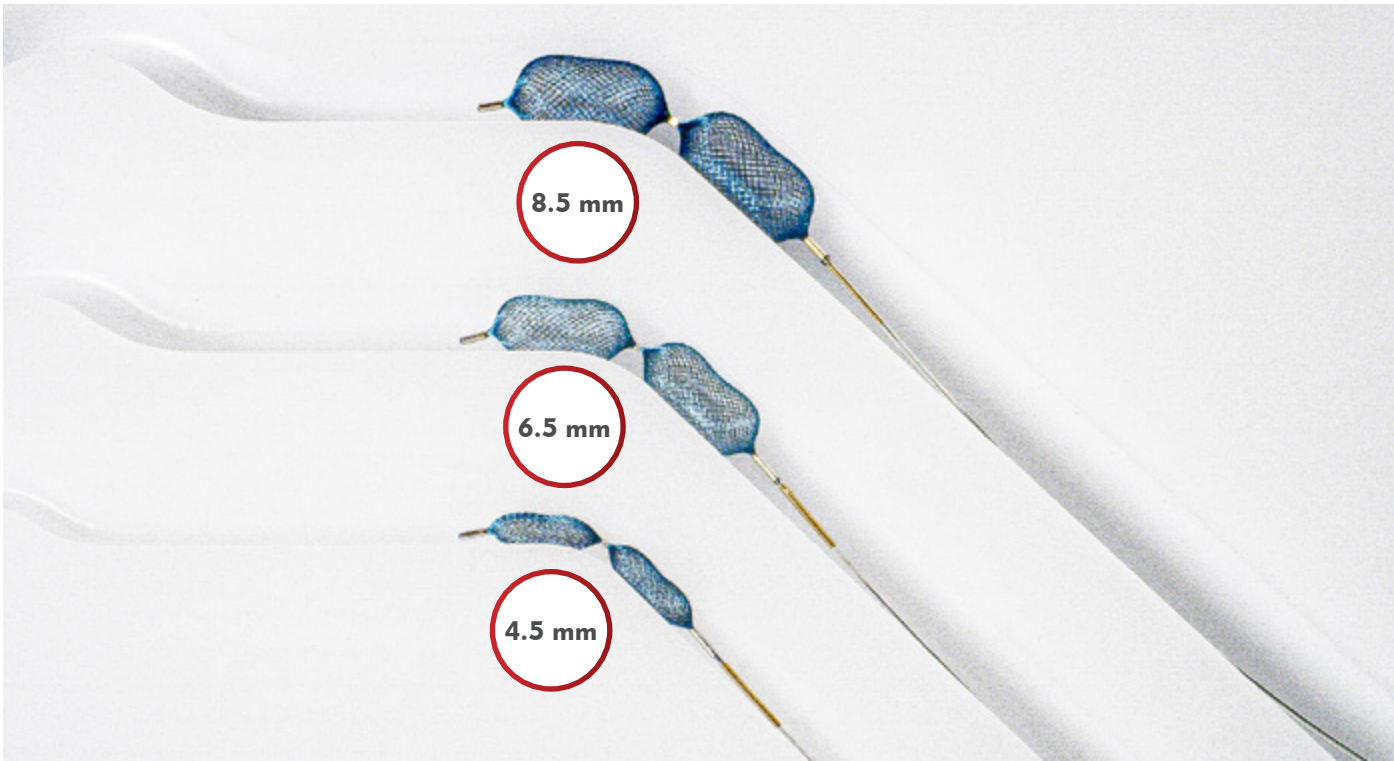

ORDERING INFORMATION

Catalog Number	Actual Plug Diameter	Minimum Artery Diameter	Maximum Artery Diameter
SVP2.5-0.021	4.5 mm	1.5 mm	2.5 mm
SVP4-0.021	6.5 mm	2.5 mm	4.0 mm
SVP6-0.027	8.5 mm	4.0 mm	6.0 mm

1. Data on File.
* Once the Siege Vascular Plug and delivery wire are separated, recapture is no longer possible.
† Has an average occlusion time of <2 minutes based on pre-clinical porcine subjects.
‡ Using like-sized microcatheters.

INDICATIONS FOR USE: The Siege Vascular Plug is indicated for arterial embolization in the peripheral vasculature.
Before using, refer to Instructions for Use for indications, contraindications, warnings, precautions, and directions for use.



Understand. Innovate. Deliver.™

merit.com

Merit Medical Systems, Inc.
1600 West Merit Parkway
South Jordan, Utah 84095
Main: +1.801.253.1600
Customer Service: +1.801.208.4300

Merit Medical Europe, Middle
East & Africa (EMEA)
Amerikalaan 42, 6199 AE
Maastricht-Airport
The Netherlands
+31 43 358 82 22

Merit Medical Ireland Ltd.
Parkmore Business Park West
Galway, Ireland
+353 (0) 91 703 733