



# Osseoflex® SN

## Steerable Needle


Vertebral augmentation using Merit’s devices is a minimally invasive procedure that combines unprecedented navigation of the vertebral body with viscous bone cement for precision treatment of symptomatic vertebral compression fractures.

Osseoflex® SN steerable, articulating needle enable direct access to the entire vertebral body via a unipedicular approach for percutaneous vertebral augmentation.

Cavity creation is achieved through the disruption of bone by steering and articulating the Osseoflex SN to create a single channel or multiple channels.

After the bone is disrupted, polymeric bone cement is injected through the Osseoflex steerable needle to fill the void.

### ACTUATION CURVE INDICATOR

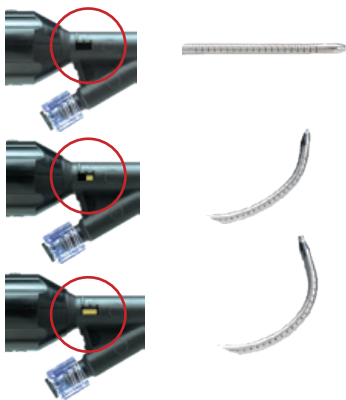

- The actuation indicator provides visual confirmation to the physician on the degree of articulation of the needle within the vertebral body.
- When partially articulated (30° - 45°), part of the yellow marker is visible in the indicator window. When fully articulated (90°) the indicator window will be completely yellow.

The ergonomically designed handle of the Osseoflex SN gives the physician control over the direction and articulation of the delivery needle.

Osseoflex steerable needle is 10 gauge compatible

### OSSEOFLEX SN STEERABLE NEEDLE DYNAMIC RANGE OF MOTION



# Preferential Path. Targeted Delivery.™

## OSSEOFLEX SN STEERABLE NEEDLE (10 GAUGE)

Part Number	Description	Contents
<b>OF-0020</b>	Osseoflex SN Steerable Needle (10 gauge)	(1) Osseoflex SN 10 gauge compatible steerable needle, (2) 10 gauge cannulas, (2) stylets, (1) bevel tip, (1) diamond tip
<b>OF-0231</b>	Osseoflex SN Steerable Needle (10 gauge) (needle only)	(1) Osseoflex SN 10 gauge compatible steerable needle
<b>OS-0001</b>	Osseoflex Access Vertebral Introduction System (10 gauge)	(2) 10 gauge cannulas, (2) stylets

As with most surgical procedures, there are risks associated with the vertebroplasty, kyphoplasty and vertebral augmentation procedures, including serious complications. For complete information regarding risks, contraindications, warnings, precautions, and adverse events please review the System's Instruction for Use.



Understand. Innovate. Deliver.™

Merit Medical Systems, Inc.  
1600 West Merit Parkway  
South Jordan, Utah 84095  
1.801.253.1600  
1.800.35.MERIT

Merit Medical Europe, Middle East, & Africa (EMEA)  
Amerikalaan 42, 6199 AE Maastricht-Airport  
The Netherlands  
+31 43 358 82 22

Merit Medical Ireland Ltd.  
Parkmore Business Park West  
Galway, Ireland  
+353 (0) 91 703 733

[merit.com](http://merit.com)

Australia  
+61 (0) 3 8373 4943

Austria  
0800 295 374

Belgium  
0800 72 906 (Dutch)  
0800 73 172 (Français)

Brazil  
+55 11 4561 - 0062

Canada  
1.800.364.4370

China  
+86-10 8561 0788

Denmark  
80 88 00 24

Finland  
0800 770 586

France  
0800 91 60 30

Germany  
0800 182 0871

Hong Kong  
+852-2207 6888

India  
+91-80-41223376

Ireland (Republic)  
1800 553 163

Italy  
800 897 005

Luxembourg  
8002 25 22

Netherlands  
0800 022 81 84

Norway  
800 11629

Portugal  
308 801 034

Russia  
+7 495 221 89 02

Singapore  
+65 67536311

South Korea  
+8210 2205 0707

Spain  
+34 911238406

Sweden  
020 792 445

Switzerland  
(Deutsch)  
+41 22 518 02 30

(Français)  
+41 22 518 02 52

(Italiano)  
+41 22 518 00 35

UK  
0800 973 115