Rad Board Xtra™

Arm Board Accessory

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

The Merit Medical Rad Board Xtra™ is a rigid PVC board designed to be used in conjunction with the Merit Medical Rad Board® (sold separately). The Rad Board Xtra is reversible, reusable, and non-sterile (Figure 1).

INDICATIONS FOR USE

The Rad Board Xtra is designed to be used in conjunction with the Rad Board to facilitate perpendicular placement of the arm for vascular access, as well as provide an additional 11” (28cm) of working space.

WARNINGS

- DO NOT sit, stand or place heavy objects on the Rad Board Xtra.
- REMOVE the Rad Board Xtra prior to moving the patient if possible. If not possible, move the patient to a surface of equal height to the table.
- If at any time the radial or brachial site is abandoned, it is recommended that the Rad Board Xtra be removed before proceeding with lower extremity access (femoral).
- DO NOT drop Rad Board Xtra as this could damage the product.

CAUTIONS

- Read instructions prior to use.
- RX Only: Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
- This device is non-sterile.
- This device is reusable.
- This device is reversible.
- This device should be used by clinicians with adequate training in the use of the device.

POTENTIAL COMPLICATIONS

NONE KNOWN

INSTRUCTIONS FOR USE

For perpendicular arm placement with the Rad Board

- Slide the Rad Board Xtra under the mattress on the procedure between the two uprights on the Rad Board (Figure 2).

NOTE: Make sure that the working part of the Rad Board Xtra is pushed all the way until the edge meets the upright at the head end of Rad Board.

NOTE: The Rad Board Xtra is reversible. When used on the right side of the table, the label is facing upward. When using on the left side of the table, the label is facing downward.

- Extend the patient’s arm onto the Rad Board Xtra. The Merit Medical Rad Rest® (sold separately) is recommended for arm placement and support during the procedure.
- After access is obtained, remove the Rad Board Xtra and place arm on the Rad Board.
- Follow instructions below if the Rad Board Xtra is to be used to provide more working space.
- After use, wipe down with a hospital approved antiseptic wipe or spray.
- Completely dry the Rad Board Xtra and store upright or flat away from extreme heat or cold.

For 11” (28cm) of additional working space

- Slide the Rad Board Xtra under the mattress on the procedure table at the foot of the Rad Board (under the sterile drape) (Figure 3).




NOTE: The Rad Board Xtra is reversible. When used on the right side of the table, the label is facing upward. When using on the left side of the table, the label is facing downward.

- Push the Rad Board Xtra until the edge is even with the Rad Board.
- After use, remove the Rad Board Xtra and wipe down with a hospital approved antiseptic wipe or spray.
- Completely dry the Rad Board Xtra and store upright or flat away from extreme heat or cold.

Manufacturer:
Merit Medical Systems, Inc., 1600 West Merit Parkway,
South Jordan, Utah, 84095 U.S.A 1-801-253-1600
U.S.A Customer service 1-800-356-3748

Authorized Representative:
Merit Medical Ireland, Ltd, Parkmore Business Park West,
Galway, Ireland

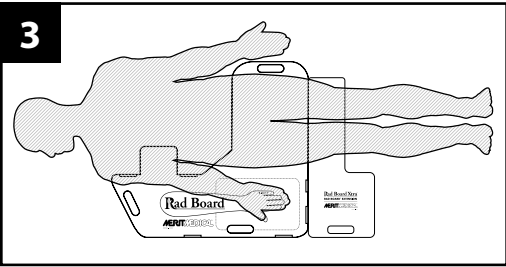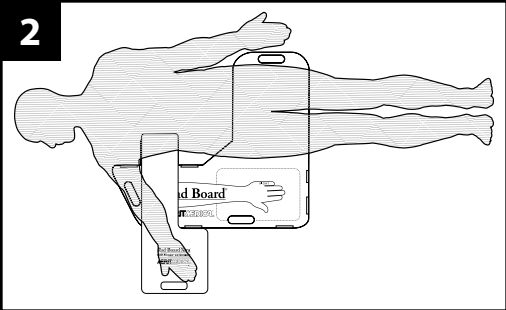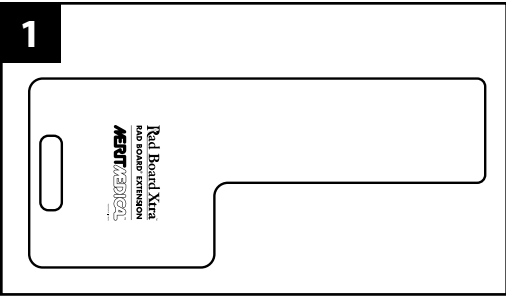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