



Arm Cushion

English

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

The Merit Medical Rad Rest® is an adjustable stabilization and positioning cushion used to hyperextend the wrist for radial access and support the forearm/elbow during procedures. Product not made with natural rubber latex (Figure 1).

INDICATIONS FOR USE

The Rad Rest is intended to be used as an adjustable upper extremity stabilization and positioning cushion when used with the Rad Board® or Rad Board Xtra™.

CLINICAL BENEFIT

- Designed to elevate the patient arm to working level, without the need for additional padding
- Patient comfort

WARNINGS

NONE KNOWN

CAUTIONS

- In the EU, any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the applicable Member State.
- Read instructions prior to use.
- RX Only: Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
- Secure the upper extremity, as needed, with tape for enhanced stabilization.
- This device is non-sterile. If used in sterile field, drape as necessary
- This device is intended for single use only. Do not reuse.
- This device should be used by clinicians with adequate training in the use of the device.

REUSE PRECAUTION STATEMENT

For single patient use only. Do not reuse or reprocess. Reuse or reprocessing may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse or reprocessing may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

POTENTIAL COMPLICATIONS

NONE KNOWN

INSTRUCTIONS FOR USE

1. Place the Rad Rest on a stable platform for upper extremity access procedures in the arm. The Merit Medical Rad Board® (sold separately) is recommended.



NOTE: The Rad Rest is non-sterile. If used in sterile field, drape as necessary.

2. Adjust the Rad Rest to fit the length of the forearm by pulling the elbow end of the Rad Rest back. Rest elbow on cushion.
3. Position the patient's forearm with the palm-side of the hand up on the Rad Rest. Tape the patient's fingers lightly to the hand end of the Rad Rest to hyperextend the wrist and secure the hand to the Rad Rest (Figure 2).
4. Continue with standard procedure and patient preparation.

NOTE: The Rad Rest is non-sterile. If used in sterile field, drape as necessary.

5. The Rad Rest is absorbent and will help to contain fluids and reduce floor runoff.
6. Dispose of Rad Rest after the procedure.

Rx Only	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Caution: Consult accompanying documents. Read instructions prior to use.
	Consult Instructions for Use
	Do not use if package is damaged.
	Non-Sterile
	Medical Device

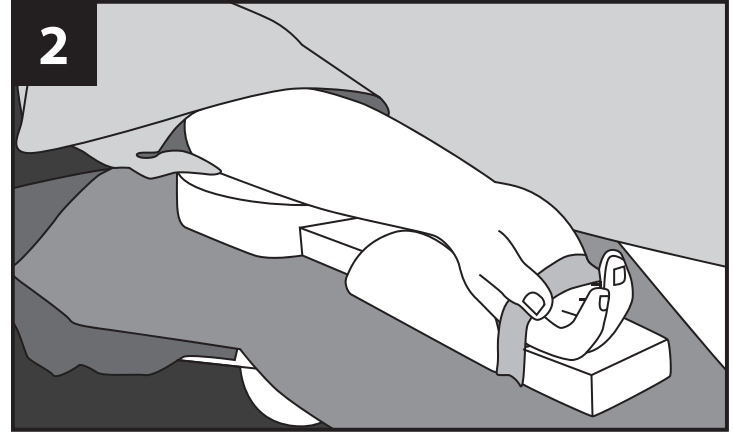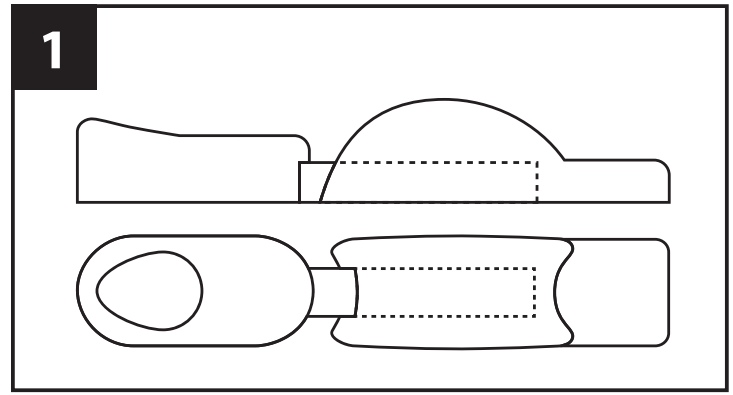

www.merit.com



Manufacturer:

Merit Medical Systems, Inc.
1600 West Merit Parkway, South Jordan, Utah 84095
U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748