ShortStop™

INSTRUCTIONS FOR USE:

DESCRIPTION:

Temporary sharps holder

INDICATIONS FOR USE:

The ShortStop temporary needle/scalpel holder is intended to prevent disease in healthcare workers by reducing the possibility of accidental sharps injury.

CAUTION:

Do not reuse. All needles should be flushed prior to use.

INSTRUCTIONS FOR USE:

Inspect the device prior to use to verify that no damage has occurred during shipping.

1. Place the ShortStop temporary sharps holder in the sterile field with the foam facing up. This can be accomplished by removing the adhesive thumb tab from the bottom, and pressing firmly to appropriate surface.

2. After use, the clinician may temporarily place the sharps device in the foam.

3. After completion of procedure, dispose of the sharps using universal precautions. Place the needle/scalpel in an OSHA compliant waste disposal system. After the sharps are removed from the foam, place the used ShortStop device in hospital compliant waste disposal system.

Device is sterilized as stated on the package label.

 MERITMEDICAL

CE 2797

Manufacturer:
Merit Medical Systems, Inc. South Jordan, Utah 84095
U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748

www.merit.com

EC REP

Authorized Representative:
Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland

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