# Mistique™

## INFUSION CATHETER

### INSTRUCTIONS FOR USE

**DESCRIPTION:** The Mistique Infusion Catheter is a 5F (1.7mm) catheter with multiple side holes.

**INDICATIONS FOR USE:** The Mistique Infusion Catheter is intended to administer infusions of various therapeutic solutions into the peripheral vasculature of a patient.

**CONTRAINDICATIONS:** The Mistique Infusion Catheter is contraindicated for use in the coronary vasculature.

The Mistique Infusion Catheter is contraindicated for use during magnetic resonance imaging.

#### CAUTIONS:

Do not use the Mistique Infusion Catheter with a power injector.

The Mistique Infusion Catheter should only be used by physicians who have a thorough understanding of infusion therapies and the associated complications of those infusion therapies.

Refer to the product insert of the therapeutic solution for indications, contraindications, cautions, and warnings.

When introducing the Mistique Infusion Catheter through a synthetic graft, an introducer sheath should be used. Damage to the infusion catheter may occur if no introducer sheath is used.

**WARNINGS:** Correct placement of the guiding wire and catheter should be verified by fluoroscopy. Failure to use fluoroscopy could result in incorrect placement resulting in patient injury or death.

A guide wire should never be advanced or withdrawn against resistance. If a guide wire is advanced where there is resistance, it could cause vessel trauma and/or wire damage. The cause of the resistance should be determined utilizing fluoroscopy.

All components must be adequately flushed with heparinized saline to displace air prior to insertion into the body.

Complications may occur if air has not been displaced.

Ensure that all connections are secure before use. Do not over tighten as excessive force may damage the product.

This device is intended for single use only.


Federal (USA) law restricts this device to sale by or on the order of a physician.

Store in a cool dry place.

Patent # 5,957,901 Other U.S. and foreign patents pending.

#### DESCRIPTION OF DEVICE:

The Mistique Infusion Catheter is a 5F (1.7mm) catheter with multiple side holes around the circumference of the catheter. The infusion portion of the catheter is identified with radiopaque marker bands to facilitate placement under fluoroscopy. The Mistique is available in a 0.035" (0.89mm) end hole design or a 0.038" (0.97mm) end hole design.

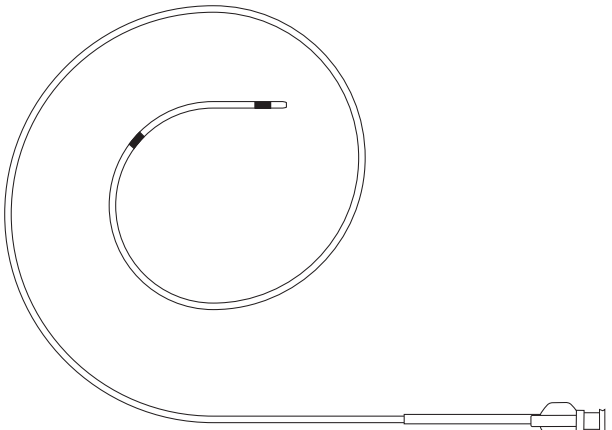#### INSTRUCTIONS FOR USE:

1. Flush the Mistique Infusion Catheter with sterile, heparinized saline to displace any air prior to placement.
2. The Mistique Infusion Catheter will pass through a standard 5 French sheath introducer.
3. Place the Mistique Infusion Catheter into position under fluoroscopic guidance over the appropriate guide wire following standard hospital protocol.
4. Leave the guide wire in place to facilitate infusion of therapeutic solution through the catheter side holes.
5. The two radiopaque marker bands on the Mistique Infusion Catheter indicate the segment where side hole infusion occurs.
6. The Mistique Infusion Catheter may be used for pulse or slow, continuous infusion of the therapeutic agent, as ordered by the physician. The rate and duration of infusion should be determined by the physician.
7. The Mistique Infusion Catheter is for one time patient use only. Do not reuse.

**MERITMEDICA®**

**CE** 2797

Manufacturer: [www.merit.com](http://www.merit.com)  
Merit Medical Systems, Inc. South Jordan, Utah 84095  
U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748

**EC REP** Authorized Representative:  
Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland  
European Customer Service by Country: Belgium 0800 72906;  
France 0800 916030; Germany 0800 1820871; Ireland 091 703700;  
Neth. 0800 0228184; U.K. 0800 973115

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