
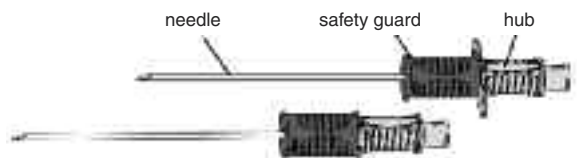
**INSTRUCTIONS FOR USE**

**INDICATIONS:**  
The SecureLoc™ Safety Introducer Needle is designed for percutaneous access. A manually activated integral safety guard reduces the risk of accidental needlestick injuries by shielding the needle sharp after use.

- WARNINGS:**
- This product is for single-use only. **DO NOT REUSE. DO NOT RESTERILIZE.**
  - Do not use if package is opened or damaged.
  - Do not alter this device.
  - Failure to fully advance the safety guard until it locks when removing the needle may result in the needle tip re-emerging from the safety guard, potentially resulting in an accidental needlestick with a contaminated needle. A needlestick with a contaminated needle may cause infectious disease.
  - Handle and discard in accordance with accepted medical practice and applicable regulations. After use, this product is a potential biohazard.

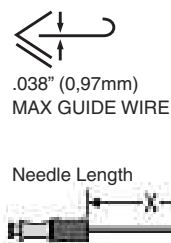- CAUTIONS:**
- Carefully read and follow all instructions prior to use.
  - Care and adherence to universal precautions must be taken to avoid accidental needlestick injuries.
  - Do not attempt to override or defeat the safety guard.
  - Only qualified healthcare practitioners should insert, manipulate and remove these devices.
  - Follow standard infection control precautions as specified by the CDC (USA) or local equivalent.
  - Federal (USA) law restricts the sale of this device to or on the order of a physician.

**INSTRUCTIONS FOR USE:**

1. Remove the needle sheath as indicated above.
2. After use, advance the safety guard to the end of the needle until you feel the safety guard stop and the safety device locks over the needle tip. Note: For guidewire applications, the safety guard may be activated before removing the needle from the guidewire.
3. Dispose of the SecureLoc™ Safety Introducer Needle in an approved sharps container while observing universal precautions.

**TRADEMARK INFORMATION:**  
SecureLoc and the SecureLoc Design are registered trademarks of Specialized Health Products®, Inc. U.S Pat #s: 6,595,955 and 6,796,962. Other patents pending.

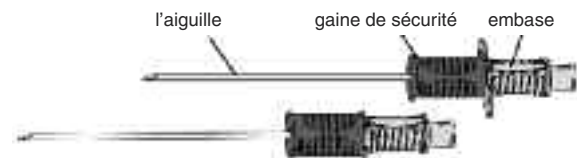
**ADDITIONAL INFORMATION:**  
An issued or revised date for these instructions is included for the users information. For additional information, the user should contact Merit Medical Systems, Inc.

**Manufacturer:**  
Merit Medical Systems, Inc. South Jordan, Utah 84095 www.merit.com  
U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748

**Authorized Representative:**  
Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland  
European Customer Service by Country: U.K. 0800 72906;  
France 0800 916030; Germany 0800 1820871; Ireland 091 703 733;  
Neth. 0800 0228184; U.K. 0800 973115 ID 102908

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