
## Hemostasis Valve Adapter

### INSTRUCTIONS FOR USE

#### PRODUCT DESCRIPTION

The Merit Hemostasis Valve Adapter consists of a hub with a swivel nut luer fitting, hemostasis valve, side arm tubing with stopcock.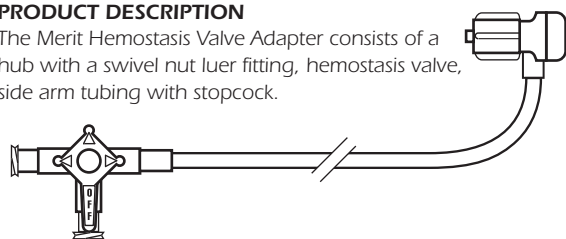

#### INTENDED USE

The intended use of the Merit Hemostasis Valve Adapter is to minimize blood loss during the introduction of catheters or devices into the vasculature, and provide a side arm and three way stopcock to facilitate aspiration or infusion.

**WARNINGS** Do not use device with power injector.

#### CAUTIONS

- Read instructions prior to use.
- Do not alter device.
- This device is intended for single use only. Do not reuse or resterilize.
- This device should only be used by physicians thoroughly trained in the technique of angiography.
- The closing force of the hemostasis valve may alter or impair the function of some catheters.
- Damage to the valve may occur if inner catheter is with drawn rapidly.
- The hemostasis valve adapter may be used with catheters that are 8F or smaller.

#### INSTRUCTIONS FOR USE

- Flush the hemostasis valve adapter, side arm and stopcock prior to use.
- Follow normal accepted practice for vessel puncture and access.
- The hemostasis valve adapter may be attached to catheter prior to catheter insertion into vessel or after the catheter is in place.
- Do not over-tighten the luer fitting of the hemostasis valve adapter.
- Follow manufacturer's recommendations for the catheter or device being introduced via the hemostasis valve adapter.
- Upon removal of the hemostasis valve adapter proper precautions should be taken to prevent bleeding.

Manufacturer:  
Merit Medical Systems, Inc. South Jordan, Utah 84095 www.merit.com  
U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748

**EC REP** Authorized Representative:

Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland  
European Customer Service by Country: Belgium 0800 72906;  
France 0800 916030; Germany 0800 1820871; Ireland 091 703700;  
Neth. 0800 0228184; U.K. 0800 973115

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