
Hemostasis Valve Adapter

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

The Merit Hemostasis Valve Adapter consists of a hub with a swivel nut luer fitting, hemostasis valve, side arm tubing with stopcock.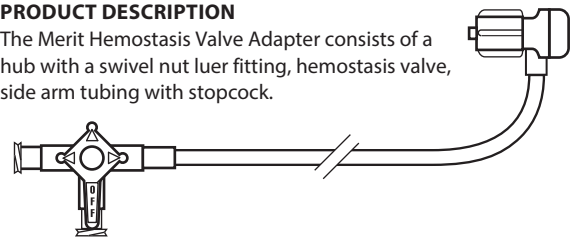

INTENDED USE

The intended use of the Merit Hemostasis Valve Adapter is to minimize blood loss during the introduction of catheters or devices into the vasculature, and provide a side arm and three way stopcock to facilitate aspiration or infusion.

WARNINGS Do not use device with power injector.

CAUTIONS

- Read instructions prior to use.
- Do not alter device.
- This device is intended for single use only. Do not reuse or resterilize.
- This device should only be used by physicians thoroughly trained in the technique of angiography.
- The closing force of the hemostasis valve may alter or impair the function of some catheters.
- Damage to the valve may occur if inner catheter is with drawn rapidly.
- The hemostasis valve adapter may be used with catheters that are 8F or smaller.

INSTRUCTIONS FOR USE

1. Flush the hemostasis valve adapter, side arm and stopcock prior to use.
2. Follow normal accepted practice for vessel puncture and access.
3. The hemostasis valve adapter may be attached to catheter prior to catheter insertion into vessel or after the catheter is in place.
4. Do not over-tighten the luer fitting of the hemostasis valve adapter.
5. Follow manufacturer's recommendations for the catheter or device being introduced via the hemostasis valve adapter.
6. Upon removal of the hemostasis valve adapter proper precautions should be taken to prevent bleeding.

Manufacturer:

Merit Medical Systems, Inc. South Jordan, Utah 84095 www.merit.com
U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748

Authorized Representative:

Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland
European Customer Service by Country: Belgium 0800 72906;
France 0800 916030; Germany 0800 1820871; Ireland 091 703700;
Neth. 0800 0228184; U.K. 0800 973115

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