English

hemostasis valve with spring

INSTRUCTIONS FOR USE

DESCRIPTION: MBAplus™ Hemostasis Valve with Spring Cap Introducer

INDICATIONS AND USAGE: The MBAplus Hemostasis Valve with spring cap introducer is indicated to maintain a fluid-tight seal around interventional and diagnostic devices during use.

CAUTION: Federal (U.S.A.) Law restricts this device to sale by or on the order of a physician.

CAUTION: The device is intended for single patient use only. Read instructions prior to use. Read manufacturer's instructions for the use of catheters, guide wires and other interventional devices.

INSTRUCTIONS FOR USE: Inspect the device prior to use to verify that no damage has occurred during shipping.

1. Connect the sideport of the hemostasis valve to the manifold assembly. To flush and fill, fully open the valve by pushing down spring cap introducer and holding. Place one finger over the male luer fitting and continue to fill the valve assembly until completely purged of air. Release the spring cap introducer to close the valve.
2. Connect the rotating male luer adapter of the hemostasis valve to the guide catheter. Ensure the connection is airtless.

CAUTION: Valve will be fully open and allow for blood loss when spring cap introducer is pushed in.

3. Push down and hold the spring cap introducer to advance interventional devices through the valve. Advance devices to intended location.
4. Disengage the spring cap introducer by releasing it. A fluid tight seal, providing little or no blood loss, will form around the interventional device while allowing movement of the guide wire, catheter or other interventional device.
5. To secure the interventional device in place, turn the spring cap introducer clockwise to tighten valve.

CAUTION: Ensure device is tightened before pressurizing. Lightly push on the spring cap introducer while turning to engage the threads.

CAUTION: Do not inject any fluid if air bubbles are visible within the valve.

NOTE: The narrowest rigid portion of the hemostasis valve has an inner diameter of 0.118 inches (0.35 cm) (approximately 9 Fr).

Manufacturer: Merit Medical Systems, Inc.
1600 West Merit Parkway, South Jordan, Utah 84095 U.S.A.
1-801-253-1600
U.S.A. Customer Service 1-800-356-3748

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