INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION
Merit Miser Contrast Management System consists of two parts:

CONTRAST SPIKE ASSEMBLY
● large bore piercing spike
● 6" large bore tubing
● large bore, 3-way, stopcock

CONTRAST BURETTE ASSEMBLY
● 2 each 1-way check valves
● 1 each 30ml flexible burette chamber
● distal tubing assembly available with or without 3-way stopcock, in small bore or large bore tubing configuration

INTENDED USE
The Merit Contrast Management System is intended for the vascular administration of contrast media and saline.

CONTRAINDICATIONS - There are no known contraindications.

WARNINGS

- Use aseptic technique.
● Do not attempt to flush contrast media or air bubbles back through the contrast delivery system when the Miser burette chamber is full of fluid.
● Flushing back through a full Miser burette chamber could damage the two check valves above the burette chamber and compromise sterility of the spike assembly and contrast container.
● Make secure connections to prevent the introduction of air into the system.
● Examine product carefully for entrapped air and fully debubble prior to injection.
● Do not use the Miser contrast burette assembly on more than one patient.
● Discard in accordance with accepted medical practice and applicable regulations.
● The Miser contrast management burette is for single patient use only.

PRECAUTIONS

- Carefully read all instructions prior to use.
● Inspect device and packaging prior to use to verify that no damage has occurred during shipping.
● Do not use the product if there are any questions concerning device sterility and/or damage.
● Insure the device is in an appropriate vertical position during use to obtain best possible clinical results.
● U.S. Federal Law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

POTENTIAL COMPLICATIONS - None Known

INSTRUCTIONS FOR USE

This device should only be used by persons qualified to conduct the procedures for which it is indicated.

CONTRAST SPIKE ASSEMBLY:

- Remove spike assembly from packaging.
● Turn stopcock handle to "off" position (pointing toward the spike).
● Remove protective cap from piercing spike head and insert into contrast container.

(Caution: The red vent door on spike should be open when used with glass bottles, and closed for plastic bags.)

CONTRAST BURETTE ASSEMBLY:

1. Attach proximal portion of the Miser burette tubing to the spike assembly stopcock.
2. Connect the fixed male luer of the distal burette tubing assembly to the manifold port.
3. Tilt burette slightly to move the orange ball to the side of the burette chamber.
4. Turn spike stopcock to "on" to allow flow from contrast container into burette.
5. To increase flow rate, create a vacuum in the burette chamber:
a. Pinch the two vent wings located at the top of the burette chamber together to open vent.
b. Squeeze chamber lightly to expel air from the burette chamber; release vent wings then release squeeze. Contrast will flow into the burette.
c. Re-open vent wings and keep open until the FILL LEVEL is reached.
6. Fill burette to FILL LEVEL indicator line on the burette. (14mL). The contrast in the burette will refill to approximately the same fill level until the contrast container source is depleted.
7. To maximize contrast conservation, the distal tubing set may be drained on models with a stopcock located beneath (distal to) the burette: Turn the 3-way stopcock on the contrast spike off to the contrast container prior to initiating the last two contrast injections.
8. Once contrast burette assembly is empty (orange ball is seated at the bottom of the burette), turn stopcock beneath burette "off" toward the burette and open "to air" via the vent filter. This allows the remaining contrast in the proximal tubing to be aspirated to the manifold.
9. At the end of the procedure, remove the Miser burette from the contrast management system and discard.
10. At the beginning of the next procedure, remove the sterile cap and attach a new Miser contrast burette assembly to the contrast spike assembly.

Do not use Miser Contrast Spike Assembly for more than 6 hours.

HOW SUPPLIED

The Miser Contrast Management System is supplied sterile in unopened, undamaged packages. These devices may be distributed in single pack pouches and in custom procedure kits.

Manufacturer: Merit Medical Systems, Inc. South Jordan, Utah 84095
U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748

Authorized Representative: Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland

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