

Merit Advance[®]

ANGIOGRAPHIC NEEDLES

INSTRUCTIONS FOR USE

INDICATION FOR USE:

The Merit Advance Angiographic Needle is indicated for providing a puncture site in blood vessels for the introduction of vascular access devices.

PRECAUTIONS:

- Read manufacturer's instructions prior to use.
- Contents of unopened, undamaged package are sterile.
- Do not use if package is damaged.
- **⚠ ONLY CAUTION:** Federal (U.S.A.) law restricts the sale of this device to sale by or on the order of a physician trained in the use of this device.
- For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

CONTRAINDICATIONS:

There are no contraindications associated with the Merit Advance Angiographic Needle.

INSTRUCTIONS FOR USE:

1. Identify the insertion site and prepare the site using proper aseptic technique and local anesthesia as required.
2. Remove the Merit Advance Angiographic Needle from package using proper aseptic technique.
3. Insert a Merit Advance Angiographic Needle using standard technique.
4. Dispose of the Merit Advance Angiographic Needle in an approved sharps container while observing universal precautions.



Do not use if package is damaged



Single Use

Single use



Non-pyrogenic



CAUTION: Consult accompanying documents



www.merit.com



Manufacturer:

Merit Medical Systems, Inc. South Jordan, Utah 84095

U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748

CE 2797



Authorized Representative:

Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland

401342001_002 ID 04-05-21