



Angiographic Catheters

INSTRUCTIONS FOR USE

DESCRIPTION:

Angiographic catheters are single-use, sterile, non-pyrogenic, disposable intravascular catheters.

INDICATIONS:

Angiographic catheters are designed to be used for delivering radiopaque media to selected sites in the vascular system in conjunction with routine diagnostic procedures. Angiographic catheters with marker bands may also be used for anatomical measurements.

CONTRAINDICATIONS:

None known.

WARNING:

- Angiographic catheters are single use.
- Recommended guide wire size: See the individual catheter label.
- Guide wire use has been associated with greater incidence of thrombus formation.

PRECAUTIONS:

- Do not use if package is opened or damaged.
- Do not use after expiration date.
- Do not autoclave.
- Carefully remove the catheters from the package by grasping the hub and slowly withdrawing it from the package.
- If resistance is felt when removing the guidewire from the catheter, remove the guide wire and catheter as a unit to prevent potential damage to the vessel wall.
- Flush all devices entering a blood vessel with sterile heparinized saline or similar isotonic solution before use.
- For catheters containing hydrophilic polymer coating on the distal surface of the catheter, the surface of the distal portion of the catheter must be completely wet to remain lubricated. Proper caution must be taken to wet the catheter prior to and during use.

ADVERSE EVENTS:

Merit Medical angiographic catheters should be used by physicians familiar with percutaneous catheter introduction. Complications which may be associated with the use of angiographic catheters include, but are not limited to, the following.

- | | |
|----------------------------------|---------------------------------------|
| • Thrombus formation/emboli | • Air emboli |
| • Hematoma at puncture site | • Arterial wall damage |
| • Plaque dislodgement | • Myocardial infarction |
| • Cardiac arrhythmias | • Infection |
| • Perforation of the vessel wall | • Allergic reaction to contrast media |
| • Vascular occlusion | • Vascular spasm |
| • Stroke and death | |

REUSE PRECAUTION STATEMENT

For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness, or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

DIRECTIONS FOR USE:

- A coated guide wire is recommended for use with Merit angiographic catheters.
- Introduce the catheter into the blood vessel using vascular entry technique of choice.
- PIGTAIL catheters. Pigtail catheters are fitted with a straightener. It is split for easy removal after the catheter tip is on the guide wire.
- Slide straightener forward until pigtail is straight.
- Place catheter tip on guide wire and remove straightener by pulling on the straightener tab.
- Marker Band: Anatomical measurement accuracy is $\pm 4\%$.

Storage: Sterile and non-pyrogenic in unopened and undamaged inner package.
Store in a cool dry place.

Shelf Life: The shelf life is established at 3 years from date of sterilization.

Transportation: Do not expose to high temperature or humidity.



Manufacturer: Merit Medical Systems, Inc.
1600 West Merit Parkway, South Jordan, Utah 84095 U.S.A.

Authorized Representative:
Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland

血管内造影导管

使用说明

产品描述:

血管内造影导管为一次性使用,灭菌,无热原的血管内导管。

适用范围:

血管内造影导管借助于日常诊断治疗,将造影介质送至血管系统中制定位置。带标记的造影导管可用于解剖学诊断。

禁忌症:

未知

警告:

- 血管内造影导管为一次性使用产品。
- 推荐导引导丝大小: 参见导管标签。
- 导引导丝的使用往往伴随着高发率的血栓形成。

注意事项:

- 包装打开或损坏的情况下,不得使用本品。
- 不得使用过期产品。
- 不要进行高压灭菌。
- 轻轻握住导管头,慢慢将导管从包装中抽出。
- 从导管抽出导引导丝时如遇到阻力,需将导引导丝和导管一同拔出,以防止破坏血管壁。
- 使用导管前,需用消毒肝素盐水或者类似等渗溶液冲洗所有进入血管内的器械。
- 对于远端表面上含亲水聚合物涂层的导管而言,导管远端部分的表面必须完全湿润,以维持润滑。使用前和使用时,必须小心谨慎地将导管弄湿。

副作用:

Merit Medical血管内造影导管只能由熟悉经皮穿刺导管介入技术的医生使用。与血管造影导管相关的并发症包括但不限于以下:

- | | | |
|-----------|---------|-----------|
| • 血栓形成/栓子 | • 空气栓塞 | • 血管闭塞 |
| • 穿刺部位血肿 | • 动脉壁损伤 | • 中风甚至死亡 |
| • 斑块脱落 | • 心肌梗死 | • 造影剂过敏反应 |
| • 心律失常 | • 感染 | • 血管痉挛 |
| • 血管壁穿孔 | | |

重复使用注意事项声明

仅可用于一名患者。请勿重复使用、重新处理或再次灭菌。重复使用、重新处理或再次灭菌可能会损坏装置的结构完整性和/或导致装置失效,而这有可能因此造成患者损伤、生病或死亡。重复使用、重新处理或再次灭菌也可能构成装置污染风险,和/或导致患者感染或交叉感染,包括但不限于在患者间传播传染病。装置污染可能导致患者受伤、患病或死亡。

使用指南:

- 建议带涂层的导引导丝与Merit血管内造影导管一起使用。
- 采用血管介入技术,将导管引入血管内。
- 猪尾式导管。猪尾式导管配有拉直器。导管尖装入导引导丝后,采取分离的方式可轻易取出。
 - 向前滑动拉直器,拉直猪尾式尾端。
 - 将导管尖装入导丝;拉动拉直器环,取出拉直器。
- 标记带: 解剖学测量的精度: $\pm 4\%$

储存条件: 本品内包装为未打开,无损坏的无菌无热原包装。存储于阴凉干燥处。

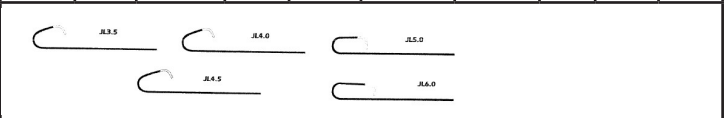
有效期: 本品从灭菌之日起有效期为3年。

运输条件: 运输过程中不应暴露在高温或高湿下。

型号、规格：见附页。

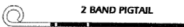
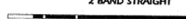
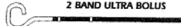
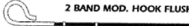
型号规格	尺寸 (F)	内径	长度	头端长度	适用导丝	描述	侧孔数量	是否带编织层	是否带标记带
Performa® Judkins 造影导管									
7501-13	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JL3.5	0	是	否
7501-21	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JL4.0	0	是	否
7501-33	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JL5.0	0	是	否
7501-43	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JL6.0	0	是	否
7501-53	6	.054"/1.37mm	125cm	1.9cm	.038" (0.97mm)	JL3.5	0	是	否
7501-63	6	.054"/1.37mm	125cm	1.9cm	.038" (0.97mm)	JL4.0	0	是	否
7501-73	6	.054"/1.37mm	125cm	1.9cm	.038" (0.97mm)	JL5.0	0	是	否
7501-83	6	.054"/1.37mm	125cm	1.9cm	.038" (0.97mm)	JL6.0	0	是	否
7501-A3	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JL3.5ST	0	是	否
7501-B1	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JL4.0ST	0	是	否
7501-C3	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JL5.0ST	0	是	否
7501-D3	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JL6.0ST	0	是	否
7501-H1	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JL4.5	0	是	否
7502-13	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JR3.5MOD	0	是	否
7502-23	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JR4.0MOD	0	是	否
7502-33	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JR5.0MOD	0	是	否
7502-43	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JR6.0MOD	0	是	否
7503-13	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JR3.5	0	是	否
7503-21	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JR4.0	0	是	否
7503-33	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JR5.0	0	是	否
7503-43	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JR6.0	0	是	否
7503-53	6	.054"/1.37mm	125cm	1.9cm	.038" (0.97mm)	JR3.5	0	是	否
7503-63	6	.054"/1.37mm	125cm	1.9cm	.038" (0.97mm)	JR4.0	0	是	否
7503-73	6	.054"/1.37mm	125cm	1.9cm	.038" (0.97mm)	JR5.0	0	是	否
7503-83	6	.054"/1.37mm	125cm	1.9cm	.038" (0.97mm)	JR6.0	0	是	否
7503-A3	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JR3.5ST	0	是	否
7503-B1	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JR4.0ST	0	是	否
7503-C3	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JR5.0ST	0	是	否
7503-D3	6	.054"/1.37mm	100cm	1.9cm	.038" (0.97mm)	JR6.0ST	0	是	否
7521-13	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JL3.5	0	是	否
7521-21	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JL4.0	0	是	否
7521-33	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JL5.0	0	是	否
7521-43	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JL6.0	0	是	否
7521-53	5	.046"/1.17mm	125cm	2.5cm	.038" (0.97mm)	JL3.5	0	是	否
7521-63	5	.046"/1.17mm	125cm	2.5cm	.038" (0.97mm)	JL4.0	0	是	否
7521-73	5	.046"/1.17mm	125cm	2.5cm	.038" (0.97mm)	JL5.0	0	是	否
7521-83	5	.046"/1.17mm	125cm	2.5cm	.038" (0.97mm)	JL6.0	0	是	否
7521-A3	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JL3.5ST	0	是	否
7521-B1	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JL4.0ST	0	是	否
7521-C3	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JL5.0ST	0	是	否
7521-F1	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JL4.5	0	是	否
7523-13	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JR3.5	0	是	否
7523-21	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JR4.0	0	是	否
7523-33	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JR5.0	0	是	否
7523-43	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JR6.0	0	是	否
7523-53	5	.046"/1.17mm	125cm	2.5cm	.038" (0.97mm)	JR3.5	0	是	否
7523-63	5	.046"/1.17mm	125cm	2.5cm	.038" (0.97mm)	JR4.0	0	是	否
7523-73	5	.046"/1.17mm	125cm	2.5cm	.038" (0.97mm)	JR5.0	0	是	否
7523-83	5	.046"/1.17mm	125cm	2.5cm	.038" (0.97mm)	JR6.0	0	是	否
7523-E3	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JR3.5ST	0	是	否
7523-F1	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JR4.0ST	0	是	否
7523-G3	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JR5.0ST	0	是	否
7523-H3	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	JR6.0ST	0	是	否
7574-13	6	.059"/1.50mm	100cm	2.5cm	.038" (0.97mm)	JL3.5	0	否	否
7574-21	6	.059"/1.50mm	100cm	2.5cm	.038" (0.97mm)	JL4.0	0	否	否
7574-33	6	.059"/1.50mm	100cm	2.5cm	.038" (0.97mm)	JL5.0	0	否	否
7574-43	6	.059"/1.50mm	100cm	2.5cm	.038" (0.97mm)	JL6.0	0	否	否

7601-13	5	.052"/1.32mm	100cm	2.5cm	.038" (0.97mm)	JL3.5	0	否	否
7601-21	5	.052"/1.32mm	100cm	2.5cm	.038" (0.97mm)	JL4.0	0	否	否
7601-33	5	.052"/1.32mm	100cm	2.5cm	.038" (0.97mm)	JL5.0	0	否	否
7601-43	5	.052"/1.32mm	100cm	2.5cm	.038" (0.97mm)	JL6.0	0	否	否
7700-10	4	.042"/1.07mm	100cm	2.3cm	.038" (0.97mm)	JL3.5	0	是	否
7700-20	4	.042"/1.07mm	100cm	2.3cm	.038" (0.97mm)	JL4.0	0	是	否
7700-30	4	.042"/1.07mm	100cm	2.3cm	.038" (0.97mm)	JL5.0	0	是	否
7700-40	4	.042"/1.07mm	100cm	2.3cm	.038" (0.97mm)	JL6.0	0	是	否
7701-10	4	.042"/1.07mm	100cm	2.5cm	.038" (0.97mm)	JR3.5	0	是	否
7701-20	4	.042"/1.07mm	100cm	2.5cm	.038" (0.97mm)	JR4.0	0	是	否
7701-30	4	.042"/1.07mm	100cm	2.5cm	.038" (0.97mm)	JR5.0	0	是	否
7701-40	4	.042"/1.07mm	100cm	2.5cm	.038" (0.97mm)	JR6.0	0	是	否



型号规格	尺寸	内径	长度	头端长度	适用导丝	描述	侧孔	是否带编织层	是否带标记带
Performa® Amplatz 造影导管									
7506-11	6	.054"/1.37mm	100cm	3.3cm	.038" (0.97mm)	ARMOD	0	是	否
7507-13	6	.054"/1.37mm	100cm	5.0cm	.038" (0.97mm)	AL1	0	是	否
7507-23	6	.054"/1.37mm	100cm	5.0cm	.038" (0.97mm)	AL2	0	是	否
7507-33	6	.054"/1.37mm	100cm	2.4cm	.038" (0.97mm)	AL3	0	是	否
7507-43	6	.054"/1.37mm	100cm	2.5cm	.038" (0.97mm)	AL4	0	是	否
7508-13	6	.054"/1.37mm	100cm	3.0cm	.038" (0.97mm)	AR1	0	是	否
7508-23	6	.054"/1.37mm	100cm	3.5cm	.038" (0.97mm)	AR2	0	是	否
7508-33	6	.054"/1.37mm	100cm	4.0cm	.038" (0.97mm)	AR3	0	是	否
7526-11	5	.046"/1.17mm	100cm	3.3cm	.038" (0.97mm)	AR MOD	0	是	否
7527-13	5	.046"/1.17mm	100cm	5.0cm	.038" (0.97mm)	AL1	0	是	否
7527-23	5	.046"/1.17mm	100cm	5.0cm	.038" (0.97mm)	AL2	0	是	否
7527-33	5	.046"/1.17mm	100cm	2.0cm	.038" (0.97mm)	AL3	0	是	否
7527-43	5	.046"/1.17mm	100cm	2.5cm	.038" (0.97mm)	AL4	0	是	否
7528-13	5	.046"/1.17mm	100cm	3.3cm	.038" (0.97mm)	AR1	0	是	否
7528-23	5	.046"/1.17mm	100cm	5.0cm	.038" (0.97mm)	AR2	0	是	否
7528-33	5	.046"/1.17mm	100cm	5.0cm	.038" (0.97mm)	AR3	0	是	否
7702-10	4	.042"/1.07mm	100cm	3.3cm	.038" (0.97mm)	AR1	0	是	否
7702-20	4	.042"/1.07mm	100cm	5.0cm	.038" (0.97mm)	AR2	0	是	否
7702-30	4	.042"/1.07mm	100cm	5.0cm	.038" (0.97mm)	AR3	0	是	否
7702-40	4	.042"/1.07mm	100cm	3.3cm	.038" (0.97mm)	AR MOD	0	是	否
7703-10	4	.042"/1.07mm	100cm	5.0cm	.038" (0.97mm)	AL1	0	是	否
7703-20	4	.042"/1.07mm	100cm	2.2cm	.038" (0.97mm)	AL2	0	是	否
7703-30	4	.042"/1.07mm	100cm	2.3cm	.038" (0.97mm)	AL3	0	是	否
7703-40	4	.042"/1.07mm	100cm	2.5cm	.038" (0.97mm)	AL4	0	是	否

型号规格	尺寸	内径	长度	头端长度	适用导丝	描述	侧孔	是否带编织层	是否带标记带
Performa® 新生儿导管									
7521-L3	5	.046"/1.17mm	70cm	2.0cm	.038" (0.97mm)	JL1.5	0	是	否
7521-M3	5	.046"/1.17mm	70cm	2.0cm	.038" (0.97mm)	JL2.0	0	是	否
7521-N3	5	.046"/1.17mm	70cm	2.0cm	.038" (0.97mm)	JL2.5	0	是	否
7521-P3	5	.046"/1.17mm	70cm	2.0cm	.038" (0.97mm)	JL3.0	0	是	否
7523-L3	5	.046"/1.17mm	70cm	2.0cm	.038" (0.97mm)	JR1.5	0	是	否
7523-M3	5	.046"/1.17mm	70cm	2.0cm	.038" (0.97mm)	JR2.0	0	是	否
7523-N3	5	.046"/1.17mm	70cm	2.0cm	.038" (0.97mm)	JR2.5	0	是	否
7523-P3	5	.046"/1.17mm	70cm	2.0cm	.038" (0.97mm)	JR3.0	0	是	否
7700-A0	4	.042"/1.07mm	70cm	2.0cm	.038" (0.97mm)	JL1.5	0	是	否
7700-B0	4	.042"/1.07mm	70cm	2.0cm	.038" (0.97mm)	JL2.0	0	是	否
7700-C0	4	.042"/1.07mm	70cm	2.0cm	.038" (0.97mm)	JL2.5	0	是	否

型号规格	尺寸	内径	长度	适用导丝	描述	侧孔	是否带 编织层	是否带 标记带	
Performa® 血管测量导管									
7602-11M	5	.052"/1.32mm	100cm	.035"(.089mm)	猪尾式	10	否	是	
7602-11M65	5	.052"/1.32mm	65cm	.035"(.089mm)	猪尾式	10	否	是	
7602-20M	5	.052"/1.32mm	100cm	.035"(.089mm)	猪尾式	10	否	是	
7602-20M100SH	5	.052"/1.32mm	100cm	.035"(.089mm)	猪尾式	5	否	是	
7602-20M65	5	.052"/1.32mm	65cm	.035"(.089mm)	猪尾式	10	否	是	
7602-20M65SH	5	.052"/1.32mm	65cm	.035"(.089mm)	猪尾式	5	否	是	
7602-2M	5	.052"/1.32mm	100cm	.035"(.089mm)	猪尾式	10	否	是	
7602-2M65	5	.052"/1.32mm	65cm	.035"(.089mm)	猪尾式	10	否	是	
7603-20M100SH	5	.052"/1.32mm	100cm	.035"(.089mm)	改良钩型	6	否	是	
7603-20M65SH	5	.052"/1.32mm	65cm	.035"(.089mm)	改良钩型	6	否	是	
7603-2M100	5	.052"/1.32mm	100cm	.035"(.089mm)	改良钩型	6	否	是	
7603-2M65	5	.052"/1.32mm	65cm	.035"(.089mm)	改良钩型	6	否	是	
7673-11M100SH	5	.052"/1.32mm	100cm	.035"(.089mm)	直式	10	否	是	
7673-11M65SH	5	.052"/1.32mm	65cm	.035"(.089mm)	直式	10	否	是	
7673-20M65SH	5	.052"/1.32mm	65cm	.035"(.089mm)	直式	10	否	是	
7673-20M100SH	5	.052"/1.32mm	100cm	.035"(.089mm)	直式	10	否	是	
7673-2M	5	.052"/1.32mm	100cm	.035"(.089mm)	直式	10	否	是	
7673-2M65	5	.052"/1.32mm	65cm	.035"(.089mm)	直式	10	否	是	
7674-2M	5	.052"/1.32mm	100cm	.035"(.089mm)	Ultra 团注	10	否	是	
7674-2M65	5	.052"/1.32mm	65cm	.035"(.089mm)	Ultra 团注	10	否	是	
7742-20M100SH	4	.041"/1.04mm	100cm	.035"(.089mm)	猪尾式	5	否	是	
7742-20M65SH	4	.041"/1.04mm	65cm	.035"(.089mm)	猪尾式	5	否	是	
2 BAND PIGTAIL 2 BAND STRAIGHT 2 BAND ULTRA BOLUS 2 BAND MOD. HOOK FLUSH									
型号规格	尺寸	内径	长度	尖端长度	适用导丝	侧孔	是否带 编织层	是否带 标记带	
Performa® 标准儿科和婴儿猪尾导管									
7756-10	4	.041"/1.04mm	65cm	6.0cm	.035"(.089mm)	5	否	否	
7756-20	4	.041"/1.04mm	80cm	6.0cm	.035"(.089mm)	5	否	否	
7757-10	4	.041"/1.04mm	65cm	5.5cm	.035"(.089mm)	4	否	否	
7757-20	4	.041"/1.04mm	80cm	5.5cm	.035"(.089mm)	4	否	否	
STANDARD PIG INFANT PIG									
型号规格	尺寸	内径	长度	尖端长度	适用导丝	描述	侧孔	是否带 编织层	是否带 标记带
Performa® 冠动脉旁路导管									
7510-13	6	.054"/1.37mm	100cm	2.5cm	.038"(.097mm)	LCB	0	是	否
7510-23	6	.054"/1.37mm	100cm	2.5cm	.038"(.097mm)	RCB	0	是	否
7534-13	5	.046"/1.17mm	100cm	2.5cm	.038"(.097mm)	LCB	0	是	否
7534-23	5	.046"/1.17mm	100cm	2.5cm	.038"(.097mm)	RCB	0	是	否
7709-10	4	.042"/1.07mm	100cm	2.5cm	.038"(.097mm)	LCB	0	是	否
7709-20	4	.042"/1.07mm	100cm	2.5cm	.038"(.097mm)	RCB	0	是	否
LCB RCB									
型号规格	尺寸	内径	长度	尖端长度	适用导丝	描述	侧孔	是否带 编织层	是否带 标记带
Performa® 升级右冠动脉导管									
7512-13	6	.054"/1.37mm	100cm	2.5cm	.038"(.097mm)	PRC	0	是	否
7707-10	4	.042"/1.07mm	100cm	2.5cm	.038"(.097mm)	PRC	0	是	否
7532-13	5	.046"/1.17mm	100cm	2.5cm	.038"(.097mm)	PRC	0	是	否
 PRC									

型号规格	尺寸	内径	长度	尖端长度	适用导丝	描述	侧孔	是否带编织层	是否带标记带
Performa® 乳膜内导管									
7511-13	6	.054"/1.37mm	100cm	3.3cm	.038"(0.97mm)	IMA	0	是	否
7531-13	5	.046"/1.17mm	100cm	3.3cm	.038"(0.97mm)	IMA	0	是	否
7706-10	4	.042"/1.07mm	100cm	3.3cm	.038"(0.97mm)	IMA	0	是	否
 IMA									

型号、压力值、粘度值为 4.7cp 条件下流量：
详见附录 B

型号	流量 (ml/s)	压力值
7501-13	20	593 psi (4089 kPa)
7501-21	20	593 psi (4089 kPa)
7501-33	20	593 psi (4089 kPa)
7501-43	20	593 psi (4089 kPa)
7501-53	18	585 psi (4033 kPa)
7501-63	18	585 psi (4033 kPa)
7501-73	18	585 psi (4033 kPa)
7501-83	18	585 psi (4033 kPa)
7501-A3	20	593 psi (4089 kPa)
7501-B1	20	593 psi (4089 kPa)
7501-C3	20	593 psi (4089 kPa)
7501-D3	20	593 psi (4089 kPa)
7501-H1	20	593 psi (4089 kPa)
7502-13	20	593 psi (4089 kPa)
7502-23	20	593 psi (4089 kPa)
7502-33	20	593 psi (4089 kPa)
7502-43	20	593 psi (4089 kPa)
7503-13	20	593 psi (4089 kPa)
7503-21	20	593 psi (4089 kPa)
7503-33	20	593 psi (4089 kPa)
7503-43	20	593 psi (4089 kPa)
7503-53	18	585 psi (4033 kPa)
7503-63	18	585 psi (4033 kPa)
7503-73	18	585 psi (4033 kPa)
7503-83	18	585 psi (4033 kPa)
7503-A3	20	593 psi (4089 kPa)
7503-B1	20	593 psi (4089 kPa)
7503-C3	20	593 psi (4089 kPa)
7503-D3	20	593 psi (4089 kPa)
7521-13	18	809 psi (5578 kPa)
7521-21	18	809 psi (5578 kPa)
7521-33	18	809 psi (5578 kPa)
7521-43	18	809 psi (5578 kPa)
7521-53	12	567 psi (3909 kPa)
7521-63	12	567 psi (3909 kPa)
7521-73	12	567 psi (3909 kPa)
7521-83	12	567 psi (3909 kPa)
7521-A3	18	567 psi (3909 kPa)
7521-B1	18	567 psi (3909 kPa)
7521-C3	18	567 psi (3909 kPa)
7521-F1	18	809 psi (5578 kPa)
7523-13	18	809 psi (5578 kPa)
7523-21	18	809 psi (5578 kPa)
7523-33	18	809 psi (5578 kPa)
7523-43	18	809 psi (5578 kPa)
7523-53	12	567 psi (3909 kPa)
7523-63	12	567 psi (3909 kPa)
7523-73	12	567 psi (3909 kPa)
7523-83	12	567 psi (3909 kPa)
7523-E3	18	809 psi (5578 kPa)
7523-F1	18	809 psi (5578 kPa)
7523-G3	18	809 psi (5578 kPa)
7523-H3	18	809 psi (5578 kPa)

7574-13	27	748 psi (5257 kPa)
7574-21	27	748 psi (5257 kPa)
7574-33	27	748 psi (5257 kPa)
7574-43	27	748 psi (5257 kPa)
7601-13	23	795 psi (5481 kPa)
7601-21	23	795 psi (5481 kPa)
7601-33	23	795 psi (5481 kPa)
7601-43	23	795 psi (5481 kPa)
7700-10	13	576 psi (3971 kPa)
7700-20	13	576 psi (3971 kPa)
7700-30	13	576 psi (3971 kPa)
7700-40	13	576 psi (3971 kPa)
7701-10	13	576 psi (3971 kPa)
7701-20	13	576 psi (3971 kPa)
7701-30	13	576 psi (3971 kPa)
7701-40	13	576 psi (3971 kPa)
7506-11	20	593 psi (4089 kPa)
7507-13	20	593 psi (4089 kPa)
7507-23	20	593 psi (4089 kPa)
7507-33	20	593 psi (4089 kPa)
7507-43	20	593 psi (4089 kPa)
7508-13	20	593 psi (4089 kPa)
7508-23	20	593 psi (4089 kPa)
7508-33	20	593 psi (4089 kPa)
7526-11	18	809 psi (5578 kPa)
7527-13	18	809 psi (5578 kPa)
7527-23	18	809 psi (5578 kPa)
7527-33	18	809 psi (5578 kPa)
7527-43	18	809 psi (5578 kPa)
7528-13	18	809 psi (5578 kPa)
7528-23	18	809 psi (5578 kPa)
7528-33	18	809 psi (5578 kPa)
7702-10	13	576 psi (3971 kPa)
7702-20	13	576 psi (3971 kPa)
7702-30	13	576 psi (3971 kPa)
7702-40	13	576 psi (3971 kPa)
7703-10	13	576 psi (3971 kPa)
7703-20	13	576 psi (3971 kPa)
7703-30	13	576 psi (3971 kPa)
7703-40	13	576 psi (3971 kPa)
7521-L3	21	868 psi (5985 kPa)
7521-M3	21	868 psi (5985 kPa)
7521-N3	21	868 psi (5985 kPa)
7521-P3	21	868 psi (5985 kPa)
7523-L3	21	868 psi (5985 kPa)
7523-M3	21	868 psi (5985 kPa)
7523-N3	21	868 psi (5985 kPa)
7523-P3	21	868 psi (5985 kPa)
7700-A0	14	611 psi (4213 kPa)
7700-B0	14	611 psi (4213 kPa)
7700-C0	14	611 psi (4213 kPa)
7700-D0	14	611 psi (4213 kPa)
7701-A0	14	611 psi (4213 kPa)
7701-B0	14	611 psi (4213 kPa)

7701-C0	14	611 psi (4213 kPa)
7701-D0	14	611 psi (4213 kPa)
7509-11	24	830 psi (5723 kPa)
7529-11	17	818 psi (5640 kPa)
7529-43	17	828 psi (5709 kPa)
7569-11	31	878 psi (6054 kPa)
7602-11	22	790 psi (5447 kPa)
7704-10	12	570 psi (3930 kPa)
7704-70	14	617 psi (4254 kPa)
7776-B1	22	790 psi (5447 kPa)
7777-B1	31	878 psi (6054 kPa)
7509-23	24	830 psi (5723 kPa)
7529-23	17	818 psi (5640 kPa)
7529-53	17	828 psi (5709 kPa)
7569-23	31	878 psi (6054 kPa)
7602-23	22	790 psi (5447 kPa)
7704-20	12	570 psi (3930 kPa)
7776-33	17	818 psi (5640 kPa)
7777-33	24	830 psi (5723 kPa)
7777-C3	31	878 psi (6054 kPa)
7529-63	17	828 psi (5709 kPa)
7602-33	22	790 psi (5447 kPa)
7776-43	17	818 psi (5640 kPa)
7513-13	21	582 psi (4013 kPa)
7513-23	20	557 psi (3840 kPa)
7513-33	21	582 psi (4013 kPa)
7513-43	20	557 psi (3840 kPa)
7513-53	24	888 psi (6123 kPa)
7513-63	24	888 psi (6123 kPa)
7513-A3	20	574 psi (3958 kPa)
7513-B3	20	593 psi (4089 kPa)
7513-C3	18	585 psi (4033 kPa)
7513-D3	20	574 psi (3958 kPa)
7513-E3	20	593 psi (4089 kPa)
7513-F3	18	585 psi (4033 kPa)
7533-13	18	809 psi (5578 kPa)
7533-23	18	837 psi (5771 kPa)
7533-33	18	809 psi (5578 kPa)
7533-43	18	837 psi (5771 kPa)
7533-53	18	744 psi (5130 kPa)
7533-63	18	744 psi (5130 kPa)
7533-B3	18	837 psi (5771 kPa)
7708-10	13	576 psi (3971 kPa)
7708-20	11	533 psi (3675 kPa)
7708-30	14	617 psi (4255 kPa)
7708-40	14	617 psi (4255 kPa)
7708-50	13	576 psi (3971 kPa)
7708-60	11	533 psi (3675 kPa)
7715-10	13	576 psi (3971 kPa)
7715-30	13	576 psi (3971 kPa)
7715-50	13	576 psi (3971 kPa)
7715-60	13	576 psi (3971 kPa)
7715-A0	13	576 psi (3971 kPa)

7715-B0	13	576 psi (3971 kPa)
7715-C0	13	576 psi (3971 kPa)
7715-D0	13	576 psi (3971 kPa)
7602-A1	35	1200 psi (8274 kPa)
7602-B1	35	1200 psi (8274 kPa)
7602-C1	35	1200 psi (8274 kPa)
7602-D1	35	1200 psi (8274 kPa)
7602-41	35	1200 psi (8274 kPa)
7602-51	35	1200 psi (8274 kPa)
7602-61	35	1200 psi (8274 kPa)
7602-71	35	1200 psi (8274 kPa)
7602-81	35	1200 psi (8274 kPa)
7602-91	35	1200 psi (8274 kPa)
7602-11M	35	1200 psi (8274 kPa)
7602-11M65	35	1200 psi (8274 kPa)
7602-20M	35	1200 psi (8274 kPa)
7602-20M100SH	35	1200 psi (8274 kPa)
7602-20M65	35	1200 psi (8274 kPa)
7602-20M65SH	35	1200 psi (8274 kPa)
7602-2M	35	1200 psi (8274 kPa)
7602-2M65	35	1200 psi (8274 kPa)
7603-20M100SH	35	1200 psi (8274 kPa)
7603-20M65SH	35	1200 psi (8274 kPa)
7603-2M100	35	1200 psi (8274 kPa)
7603-2M65	35	1200 psi (8274 kPa)
7673-11M100SH	35	1200 psi (8274 kPa)
7673-11M65SH	35	1200 psi (8274 kPa)
7673-20M65SH	35	1200 psi (8274 kPa)
7673-20M100SH	35	1200 psi (8274 kPa)
7673-2M	35	1200 psi (8274 kPa)
7673-2M65	35	1200 psi (8274 kPa)
7674-2M	35	1200 psi (8274 kPa)
7674-2M65	35	1200 psi (8274 kPa)
7742-20M100SH	35	1200 psi (8274 kPa)
7742-20M65SH	35	1200 psi (8274 kPa)
7756-10	15	583 psi (4020 kPa)
7756-20	13	584 psi (4027 kPa)
7757-10	15	583 psi (4020 kPa)
7757-20	13	584 psi (4027 kPa)
7510-13	20	593 psi (4089 kPa)
7510-23	20	593 psi (4089 kPa)
7534-13	18	809 psi (5578 kPa)
7534-23	18	809 psi (5578 kPa)
7709-10	13	576 psi (3971 kPa)
7709-20	13	576 psi (3971 kPa)
7512-13	20	593 psi (4089 kPa)
7707-10	12	576 psi (3971 kPa)
7532-13	18	809 psi (5578 kPa)
7511-13	20	593 psi (4089 kPa)
7531-13	18	809 psi (5578 kPa)
7706-10	12	576 psi (3971 kPa)

注：无侧孔和侧孔仅在头端的型号，压力值为头端夹闭情况下测试值。

产品名称：血管内造影导管
结构及组成：血管内造影导管包括Perfoma造影导管。产品由导管座（聚碳酸酯；氧化钛），导管轴（尼龙11; PEBAX;硫酸钡），编织层（304V不锈钢丝），导管尖端（PEBAX, 氧化钛，次碳酸铋）及应力缓冲套管（尼龙11, PEBAX, 氧化钛）组成，远端带有不透射线标记（铂-铱）。环氧乙烷灭菌，一次性使用。
生产日期和失效日期：见产品标签
注册证编号：国械注进20153031099
产品技术要求编号：国械注进20153031099
中国境内代理人及售后服务单位名称：麦瑞通医疗器械（北京）有限公司
中国境内代理人及售后服务单位住所：北京市朝阳区东大桥路9号楼2单元801室内B01、B02及B03单元
中国境内代理人及售后服务单位电话：010-85610788
中国境内代理人及售后服务单位传真：010-85616981
注册人及生产企业名称：MERIT MEDICAL SYSTEMS, INC. 美国麦瑞通医疗设备有限公司
注册人及生产企业住所：1600 West Merit Pkwy South Jordan, UT 84095 USA
生产地址:1) 14646 Kirby Drive Houston Texas 77047 USA;
2) Avenida Sor Juana Inés de la Cruz 19970 interior B, Edificio 2 Parque Industrial Frontera Tijuana, Baja California C.P. 22630 Mexico
注册人及生产企业联系方式：1-801-253-1600
说明书编制或修订日期：2024年9月