
INSTRUCTIONS FOR USE

DESCRIPTION:

The Merit basixCOMPAK Inflation Syringe is a 20ml disposable device fitted with a threaded plunger and lock / release bar, capable of producing a maximum pressure of 30 ATM/BAR, a flexible high pressure extension tube, and a three-way medium pressure stopcock. The basixCOMPAK is designed to generate and monitor pressures over a range of -1 to +30 ATM/BAR (-14.7 to +441 PSI) with accuracy of ±1 ATM/BAR (14.7 PSI).

PRODUCT INTRODUCTION:

The product includes a series of models based upon operational purpose. The model name and components for each are as follows:

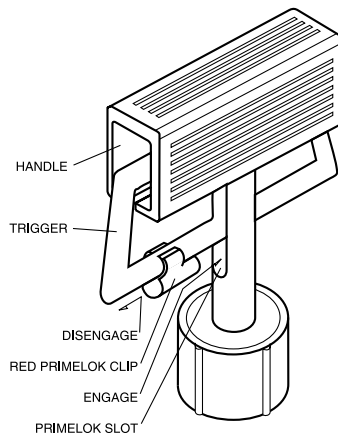IN4130: 30 ATM analog inflation device with 13" (35.5cm) extension tubing and angled gauge, 3-way stopcock included. Packaged 5 units per box, 4 boxes per case.

IN4230: 30 ATM analog inflation device with angled gauge, 13" (35.5cm) extension tubing, Access-9™ and Merit Angioplasty Pack™ (MAP). MAP includes large bore hemostasis valve, metal guide wire insertion tool and torque device. 3-way stopcock included. Packaged 5 kits per box, 4 boxes per case.

IN4530: 30 ATM analog inflation device with angled gauge, 13" (35.5cm) extension tubing, AccessPlus™ and Merit Angioplasty Pack™ (MAP). MAP includes large bore hemostasis valve, metal guide wire insertion tool and torque device. 3-way stopcock included. Packaged 5 kits per box, 4 boxes per case.

IN4430: 30 ATM analog inflation device with angled gauge, 13" (35.5cm) extension tubing, MBA™ and Merit Angioplasty Pack™ (MAP). MAP includes large bore hemostasis valve, metal guide wire insertion tool and torque device. 3-way stopcock included. Packaged 5 kits per box, 4 boxes per case.

IN4330: 30 ATM analog inflation device with angled gauge, 13" (35.5cm) extension tubing, Honor® and Merit Angioplasty Pack™ (MAP). MAP includes large bore hemostasis valve, metal guide wire insertion tool and torque device. 3-way stopcock included. Packaged 5 kits per box, 4 boxes per case.

IN4352: 30 ATM analog inflation device with angled gauge, 13" (35.5cm) extension tubing, Honor® (20cm tubing extension) and Merit Angioplasty Pack™ (MAP). MAP includes large bore hemostasis valve, metal guide wire insertion tool and torque device. 3-way stopcock included. Packaged 5 kits per box, 4 boxes per case.

K05-00807: 30 ATM analog inflation device with 13" (35.5cm) extension tubing and with 10ml Red Nitro & 10ml Yellow Heparin Medallion® Syringes. 3-way stopcock included. Packaged 5 kits per box, 4 boxes per case.

K05-01840: 30 ATM analog inflation device with angled gauge, 13" (35.5cm) extension tubing and Merit Angioplasty Pack™ (MAP). MAP includes the AccessPlus™ large bore hemostasis valve, metal guide wire insertion tool, torque device and RXP® - rapid exchange prep syringe. 3-way stopcock included. Packaged 5 kits per box, 4 boxes per case.

K05-00812: 30 ATM analog inflation device with 13" (35.5cm) extension tubing and 20ml VacLok® syringe. 3-way stopcock included. Packaged 5 kits per box, 4 boxes per case.

K05-00892: 30 ATM analog inflation device with 13" (35.5cm) extension tubing, includes Access-9™ large bore hemostasis valve. 3-way stopcock included. Packaged 5 kits per box, 4 boxes per case.

K05-00808: 30 ATM analog inflation device with 13" (35.5cm) extension tubing and 10ml red NITRO Medallion® syringe. 3-way stopcock included. Packaged 5 kits per box, 4 boxes per case.

K05-00842: 30 ATM analog inflation device with 13" (35.5cm) extension tubing, Merit Angioplasty Pack™ (MAP) and 10ml red NITRO Medallion® syringe. MAP includes Access-9™ large bore hemostasis valve, metal guide wire insertion tool and torque device. 3-way stopcock included. Packaged 5 kits per box, 4 boxes per case.

INDICATIONS AND USAGE:

This inflation device is used to inflate and deflate an angioplasty balloon or other interventional device, and to measure the pressure within the balloon.

INSTRUCTIONS FOR USE:

Before use, inspect the device and packaging to verify that no damage has occurred as a result of shipping.

PRIMING WITH PRIMELOK™:

The PrimeLok allows purging of air and excess fluid without squeezing the trigger. The basixCOMPAK is packaged with the PrimeLok and with the lock/release mechanism in the disengaged position.

1. To engage PrimeLok, squeeze trigger and slide PrimeLok into slot in plunger.

CAUTION: RX Only – Federal (USA) law restricts this device to sale by or on the order of a physician.

2. To prep syringe, simply aspirate up to 20ml of contrast solution into the inflation syringe by pulling back on the handle.

CAUTION: Inspect the syringe tubing and stopcock (if used) to insure that there is no air in the system.

3. Push handle against table to remove air in syringe.

4. To disengage PrimeLok, squeeze handle and slide PrimeLok out of slot. This will allow the trigger to lock in position and the device is now ready for use.

CAUTION: The PrimeLok must be disengaged before pressure can be maintained by the lock/release mechanism.

ATTACHING THE INFLATION DEVICE TO THE BALLOON:

NOTE: Refer to the manufacturer's directions accompanying the balloon dilatation catheter or other interventional device for specific information on use, maximum inflation pressure, precautions, and warnings for that device.

1. Prepare and test the balloon catheter according to the catheter manufacturer's directions for use.

2. Create a fluid-fluid connection between the balloon and the inflation syringe extension tube and connect the luer connectors securely.

3. Squeeze the trigger and pull back on the handle to apply a vacuum to the balloon.

BALLOON INFLATION AND DEFLATION:

1. To inflate the balloon, squeeze the trigger allowing the plunger to return to the resting position (0 ATM/BAR). Release grip on the trigger, locking the plunger into position. To increase pressure, rotate the handle clockwise until the desired inflation pressure is reached. The lock mechanism maintains the pressure.

NOTE: Loss of pressure may indicate a leak in the system.

CAUTION: To protect the threads of the lock release handle, the device must be displaying 25 ATM or lower before the quick release mechanism is used to deflate the angioplasty balloon.




2. To deflate the balloon, squeeze the trigger and pull back to generate a negative pressure. Release grip on the trigger to lock the plunger in the negative pressure position.

Shelf Life:	30 Months
Storage Conditions:	Well-Ventilated, Dry, Room Temperature
Shipping Conditions:	Refer to Shipping Contract
Certificate No:	SFDA (I) 20112210006
Registration Standards No.:	YZB/USA 3145-2010
Name of After Sales	
Service Provider:	Jyton & Emergo (Beijing) Medical Technology Co., Ltd.

Manufacturer:
Merit Medical Systems, Inc.
1600 West Merit Parkway, South Jordan, Utah 84095 U.S.A.

Authorized Representative:
Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland
401961001/B ID 051911

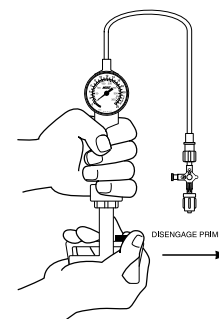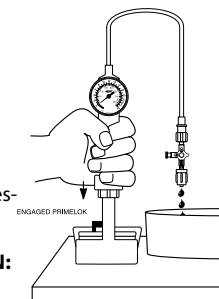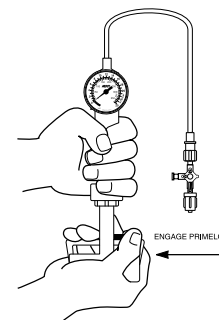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