# Merit CCS<sup>TM</sup> Coronary Control Syringes

## INSTRUCTIONS FOR USE

### DESCRIPTION:

Merit CCS Coronary Control Syringes

### CAUTION:

Read instructions prior to use.

**Rx Only:** U.S. Federal law restricts this device to sale by or on the order of a physician.

Inspect the device prior to use to verify that no damage has occurred during shipping.

This device is intended for single patient use only.

### Reuse Precaution Statement:

For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

### Product Performance Structure and Composition:

The coronary control syringe is composed of a syringe barrel, seal/tip, head (connector), plunger, handle, o-ring and rotating adapter (rotator configurations only), with different types of handles and rings.

### Intended Use:

The Merit Coronary Control Syringe is intended to be used by a cardiologist or radiologist during angiographic or radiologic procedures.

### Instructions for use:

1. Remove the syringe from the package.
2. To prep the syringe, using one or two hands, pump the piston back and forth several times to ensure that the piston moves smoothly.
3. To fill the syringe, pull plunger back.
4. Connect the syringe head to the female luer by turning the syringe clockwise
5. To inject, push the syringe plunger, injecting the appropriate amount of medication.

### Note:

In an unopened or undamaged condition, the product is disposable, sterilized, aseptic, non-pyrogenic, and for single use only. Do not use the product if the package is found opened or damaged.

Shelf life: 3 years.

Storage Conditions: Store in a cool, dry place.

Transportation Conditions: Do not expose to excessive heat or humidity during transportation.

Manufacturer: Merit Medical Systems, Inc. 1600 West Merit Parkway, South Jordan, Utah 84095  
U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748

[www.merit.com](http://www.merit.com)

Authorized Representative: Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland +31 43 358 82 22

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