MiniStop® Filter

INSTRUCTIONS FOR USE


PRODUCT DESCRIPTION:
MiniStop™ waste basin with lid and filter

INTENDED USE
The MiniStop Filter Disposal System is intended for the collection and disposal of blood and other fluids collected during invasive procedures. The device includes a filter basket for the collection of emboli or thrombi larger than 70 µm.

- PRECAUTIONS**
- Read instructions prior to use.
 - Inspect contents prior to use.

REUSE PRECAUTION STATEMENT
For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

- INSTRUCTION FOR USE**
- Remove the filter basket from the center of the waste basin of the MiniStop Filter Disposal System.
 - Wet the mesh of the filter basket to allow fluid flow.
 - Return the filter basket to the center of the MiniStop waste basin.
 - Carefully dispense extracted blood and/or other fluids into filter basket (blood and/or other fluids will go through filter basket and collect in MiniStop) and filter the extracted fluids through the filter basket. Any fresh, soft emboli and/or thrombi (larger than 70 micron) that have been aspirated should remain in the filter basket.
 - Specimen in filter is ready for lab analysis. If necessary, carefully remove filter with clot and/or other specimen for laboratory analysis per hospital protocol.
 - Locate the lid for the MiniStop Filter Disposal System underneath the MiniStop waste basin.
 - Place the lid on the MiniStop waste basin until lid snaps into place.
 - Dispose of MiniStop Filter Disposal System according to designated guidelines for the disposal of contaminated human waste.

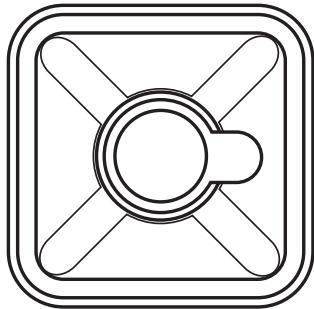Rx ONLY	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Single use
	Caution: Consult accompanying documents. Read instructions prior to use.
	Do not use if package is damaged

Ministop is a registered and unregistered trademark of Merit Medical Systems, Inc.

Manufacturer:
Merit Medical Systems, Inc.
1600 West Merit Parkway, South Jordan, Utah 84095 U.S.A.
1-801-253-1600
U.S.A Customer Service 1-800-356-3748

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