

# MEDALLION IZOFF™ SYRINGE

## INSTRUCTIONS FOR USE

### DESCRIPTION

The Medallion IZOff™ Syringe, is a general piston syringe constructed using a barrel, ratcheted plunger, piston seal and milliliter increments.

### INDICATIONS FOR USE

The Merit Syringe is used to inject fluids into, or withdraw fluids from, the body.

### CAUTIONS

- Rx Only Caution: Federal (USA) law restricts this device to sale by or on the order of a physician trained and/or experienced in the use of the device.
- Inspect the device prior to use to verify that no damage has occurred during shipping.
- For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or reesterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or reesterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient.
- The incremental mechanism is designed to inject 1 mL increments. When withdrawing fluid, refer to the print scale for volume accuracy.
- The incremental mechanism employs tactile and audible feedback to advance the plunger. It is recommended to familiarize oneself with the tactile and audible feedback with an empty syringe prior to use to become familiar with the amount of force it takes to advance the syringe.
- Injecting through a small diameter device or with viscous fluids may affect the force feedback from the incremental mechanism.
- Not for Medication Storage.

### CONTRAINDICATION










None known.

### INSTRUCTIONS FOR USE

1. Using aseptic technique, open the package containing the sterile product.
2. Inspect for damage or improper assembly.
3. To aspirate fluids, pull plunger back and maintain pressure.
4. To inject, push plunger forward. Note: One (1) audible click and tactile feel will help indicate when 1 mL has been injected.

Nominal capacity accuracy =  $\pm 4\%$ .

### TABLE OF SYMBOLS

	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Caution: Consult accompanying documents. Read instructions prior to use.
	Do Not Reuse.
	Do not reesterilize
	Non-pyrogenic
	Do not use if package is damaged
	Sterilized using ethylene oxide
	Use by date: year-month-day
	For electronic copy scan QR code, or go to <a href="http://www.merit.com/ifu">www.merit.com/ifu</a> and enter IFU ID. For printed copy, call U.S.A. or EU Customer Service



[www.merit.com](http://www.merit.com)



Manufacturer:  
Merit Medical Systems, Inc.  
1600 West Merit Parkway  
South Jordan, Utah 84095 U.S.A.  
1-801-253-1600  
U.S.A. Customer Service 1-800-356-3748



Authorized Representative:  
Merit Medical Ireland Ltd  
Parkmore Business Park West, Galway, Ireland  
EC Customer Service +31 43 3588222

403430002EN\_002 ID 2020-01-08