



Coronary Control Syringe

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

The Merit Coronary Control Syringe contains a calibrated hollow barrel into which is inserted a closely fitted movable plunger and tip or seal. The barrel contains a fixed male luer connector or male luer rotating adaptor, which are compatible with female luer hubs.

INDICATIONS FOR USE

The Merit Coronary Control Syringe is intended to be used for the intra-arterial or intravenous administration of radiographic contrast media. The syringe can also be used to inject fluids into, or withdraw fluids from, the body.

CLINICAL BENEFITS

Administration or withdrawal of fluids or the creation of a vacuum during appropriate procedures.

CONTRAINDICATIONS

None known

PRECAUTIONS

Carefully read instructions before using product. If product is being used in conjunction with other manufacturers' components, also read their Instructions for Use.

Use Proper aseptic techniques while handling product.

Rx ONLY Caution: Federal (USA) law restricts this device to sale by or on the order of a physician trained and/or experienced in the use of the device.

Nominal capacity accuracy per ISO 7886-1

REUSE PRECAUTION STATEMENT

For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

INSTRUCTIONS FOR USE

1. Using aseptic technique, open the package containing the sterile product.
2. Inspect the device prior to use to verify that no damage has occurred during shipping.
3. To prep the syringe, using one or two hands, pump the piston back and forth several times to ensure that the piston moves smoothly.
4. To fill the syringe, pull plunger back.
5. Connect the syringe head to the female luer by turning the syringe clockwise. DO NOT OVER-TIGHTEN. Do not use any instrument to tighten connections.
6. To inject, push the syringe plunger, injecting the appropriate amount of fluid.

Storage Conditions: Store in a cool, dry place.

Transportation Conditions: Do not expose to excessive heat or humidity during transportation.

SYMBOL	DESIGNATION
	Single use
	Caution
	Non-pyrogenic
	Do Not Use If Package is Damaged and Consult Instruction for Use
	Manufacturer
	Use by date: YYYY-MM-DD
	Catalog number
	Lot number
	Date of Manufacture: YYYY-MM-DD
	Medical Device
	Unique Device Identifier
	Consult Instructions for Use For electronic copy scan QR code, or go to www.merit.com/ifu and enter IFU ID. For printed copy, call U.S.A. or EU Customer Service
	Do not resterilize
	Single sterile barrier system
	Sterilized Using Ethylene Oxide
	Sterilized Using Irradiation



Manufacturer:
Merit Medical Systems, Inc.
1600 West Merit Parkway
South Jordan, Utah 84095 U.S.A.
1-801-253-1600
U.S.A. Customer Service 1-800-356-3748