**DualCap®**

IV Pole Strips®

Disinfecting Caps for Needle-free Valves

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**INSTRUCTIONS FOR USE**

Each sterile DualCap® contains 70% isopropyl alcohol.

**Use Aseptic Technique****DUALCAP INSTALLATION**

1. Grasp the light blue cap and peel to remove from the IV Pole Strip®.
2. Twist the light blue cap onto the needleless luer access valve in clockwise direction.

**TO REMOVE**

1. Using two hands, grasp the needleless luer access valve and twist the light blue cap counter clockwise.
2. Visually inspect the needleless luer access valve to verify that it is securely attached to the catheter.
3. Discard DualCap per institutional guidelines after use.

No swabbing with alcohol is necessary when you keep all luer access valves capped with a light blue DualCap.

A new light blue DualCap should be placed on all luer access valves each time the IV line is disconnected and not in use.

**INDICATIONS FOR USE**

When left in place for 30 seconds, the Light Blue DualCap disinfects needleless luer access valves; thereafter the caps provide a physical barrier to contamination up to 7 days under normal conditions if not removed.

**WARNINGS**

- Do not use if package is broken, torn or punctured, or if the sponge appears to be dry.
- Do not re-use. Do not resterilize.
- Follow instructions for use and institutional guidelines for disposal.
- Two-handed removal must be used to ensure luer access valve is not removed unintentionally.

This device is NOT made with Natural Rubber Latex.

	Caution, Consult Instructions for Use
	Caution
	Single Use. Do not reuse.
	Do not resterilize
	Use by date
	Catalog Number
	Sterilized using Irradiation
	Batch Code
	Temperature limitation
	Federal (U.S.A.) Law restricts this device to sale by or on the order of a physician
	Do not use if package is damaged

**1. SEPARATE****2. CONNECT****3. DISINFECT**

2797

www.merit.com

Manufacturer:  
Merit Medical Systems, Inc.  
1600 West Merit Parkway, South Jordan, Utah  
84095 U.S.A. 1-801-253-1600 U.S.A.  
Customer Service 1-800-356-3748

[EC REP] Authorized Representative:  
Merit Medical Ireland Ltd,  
Parkmore Business Park West, Galway, Ireland  
EC Customer Service +31 43 358822

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