# TEMNO

## *Evolution*®

EN	INSTRUCTIONS FOR USE	2
FR	MODE D'EMPLOI	3
IT	ISTRUZIONI PER L'USO	4
DE	GEBRAUCHSANWEISUNG	5
ES	INSTRUCCIONES DE USO	6
PT	INSTRUÇÕES DE UTILIZAÇÃO	7
PTBR	INSTRUÇÕES DE USO	8
NL	GEBRUIKSINSTRUCTIES	9
SV	BRUKSANVISNING	10
DA	BRUGSANVISNING	11
EL	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	12
TR	KULLANMA TALİMATLARI	13
PL	INSTRUKCJA UŻYTKOWANIA	14
CS	NÁVOD K POUŽITÍ	15
BG	ИНСТРУКЦИИ ЗА УПОТРЕБА	16
HU	HASZNÁLATI ÚTMUTATÓ	17
RU	РУКОВОДСТВО ПО ЭКСПЛУАТАЦИИ	18
FI	KÄYTTÖOHJEET	19
ZH	使用说明	20
JA	使用説明	21

**INSTRUCTIONS FOR USE****DEVICE DESCRIPTION**

The TEMNO Evolution<sup>®</sup> biopsy system is a single patient use core biopsy device. It is available in several needle gauge sizes and lengths. The plunger is color coded according to the various gauge sizes (e.g., green=20G, orange=18G, blue=16G, white=14G).

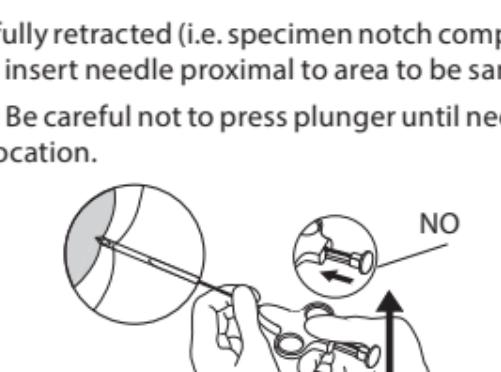
**INDICATIONS FOR USE**

The TEMNO Evolution biopsy device is used to remove, by cutting, a specimen of tissue for microscopic evaluation.

**STEP 1**

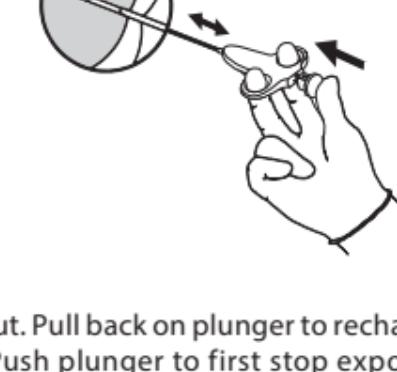
Charge TEMNO needle by pulling back on plunger until firm click is felt indicating that the needle spring is charged. It is possible to load the needle in two different positions.

- First Click = 10 mm Notch Exposure
- Second Click = 20 mm Notch Exposure

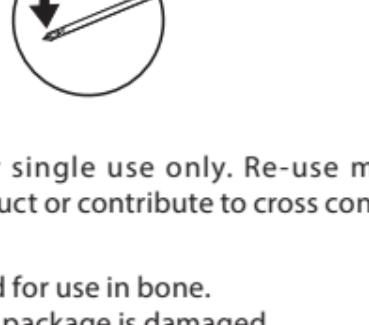
**STEP 2**

With stylet fully retracted (i.e. specimen notch completely covered by cannula) insert needle proximal to area to be sampled.

**WARNING:** Be careful not to press plunger until needle is inserted to desired location.

**STEP 3**

Press plunger with thumb to advance stylet exposing specimen notch. Press plunger past stop with thumb, automatically triggering the cannula to close over the notch thereby trapping the specimen.

**CAUTION:** For single use only. Re-use may result in a non-functional product or contribute to cross contamination.

**WARNING**

- Not intended for use in bone.
- Do not use if package is damaged.

Only clinicians who have been properly trained in this procedure, including patient preparation, sample preservation and the use of image guidance for percutaneous biopsy, should use this device.

SYMBOL	DESIGNATION
	Use By: YYYY-MM-DD
	Lot Number
	Catalog Number
	Sterilized Using Ethylene Oxide
	Do Not Use if Package is Damaged
	Single Use
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician
	Caution: Consult accompanying documents

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Merit Medical Systems, Inc.
South Jordan, Utah 84095 U.S.A.
U.S.A Customer Service 1-800-356-3748
Merit Medical Ireland Ltd,