INSTRUCTIONS FOR USE


Intended for use in obtaining core biopsy samples from soft tissue such as breast, kidney, liver, lung, thyroid, lymph nodes and various soft tissue masses.

Not intended for use in bone.

Do not use if package is damaged.

STEP 1

Charge Temno needle by pulling back on plunger until firm click is felt indicating that the needle spring is charged. It is possible to load the needle in two different positions.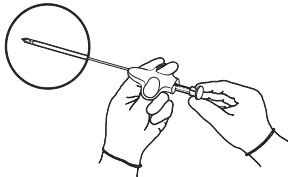


- FIRST CLICK = 10 MM NOTCH EXPOSURE
- SECOND CLICK = 20 MM NOTCH EXPOSURE

STEP 2


With stylet fully retracted (i.e. specimen notch completely covered by cannula as in Fig. 2) insert needle proximal to area to be sampled.

WARNING: Be careful not to press plunger until needle is inserted to desired location.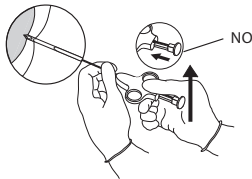


STEP 3

Press plunger with thumb to advance stylet exposing specimen notch. Press plunger past stop with thumb, automatically triggering the cannula to close over the notch thereby trapping the specimen.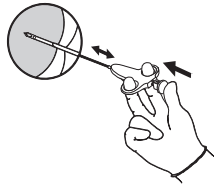

STEP 4: FIRE THE INSTRUMENT

Draw needle out. Pull back on plunger to recharge the spring until second click. Push plunger to first stop exposing the specimen notch. Remove tissue specimen from notch in stylet. **Be careful not to push plunger past first stop during this step.**

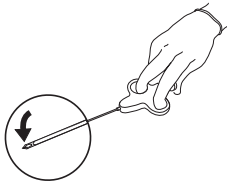CAUTION: For single use only. Re-use may result in a non-functional product or contribute to cross contamination.

Only clinicians who have been properly trained in this procedure, including patient preparation, sample preservation and the use of image guidance for percutaneous biopsy, should use this device.

	Use By: YYYY-MM-DD
	Lot Number
	Catalog Number
	Sterilized Using Ethylene Oxide
	Do Not Use if Package is Damaged
	Single Use
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician
	For electronic copy scan QR code or go to www.merit.com/ifu and enter IFU ID Number. For printed copy, call U.S.A or E.U Customer Service.
	Medical Device
Sterile Package	Sterile Package
	Manufacturer
	Date of Manufacture: YYYY-MM-DD

www.merit.com

Manufacturer:
Merit Medical Systems, Inc.
1600 West Merit Parkway,
South Jordan, Utah 84095 U.S.A.
1-801-253-1600
U.S.A Customer Service 1-800-356-3748

Authorized Representative:
Merit Medical Ireland Ltd,
Parkmore Business Park West,
Galway, Ireland
EU Customer Service +31 43 358 82 22

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