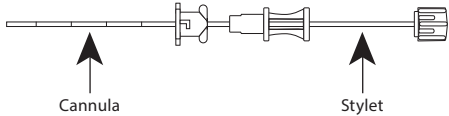
Coaxial Introducer Needle

INSTRUCTIONS FOR USE

Intended for use in soft tissue biopsies such as breast, kidney, liver, lung, prostate, thyroid, lymph nodes, and various soft tissue masses.

Not intended for use in bone.

Do not use if package is damaged.

NOTE: Not intended for use in bone. Not compatible with MRI.

CAUTION: Use with imaging equipment such as ultrasound, X-ray, CT, etc.

INSTRUCTIONS FOR USE

- During the pre-procedure set-up, insert the biopsy device through the coaxial introducer needle to verify compatibility.
- Inspect the needle for damage before use.
- Adjust sliding skin stopper on coaxial introducer needle to desired calibration and lock in place.
- Insert introducer needle until sliding stopper rests on skin surface. Image coaxial needle, if desired, to make sure the tip is at or near the edge of the region to be sampled.
- Pull the stylet out of the cannula.
- Insert a procedural biopsy needle into the introducer needle until the biopsy device touches the hub on the introducer needle.
NOTE: Automated devices must be prepared to fire prior to insertion.
- Perform the biopsy.
- Withdraw the biopsy needle from the introducer needle and remove the specimen.
- If another biopsy of the same tissue is required, repeat steps 6 and 7.
NOTE: If changing the angle of the introducer needle is required, reinsert the stylet prior to repositioning the needle. Remove the stylet prior to taking the biopsy.
- Remove the introducer needle from biopsy site.

CAUTION: For single use only. Re-use may result in a non-functional product or contribute to cross contamination.

Only clinicians who have been properly trained in this procedure, including patient preparation, sample preservation and the use of image guidance for percutaneous biopsy, should use this device.

	Use By: YYYY-MM-DD
	Lot Number
	Catalog Number
	Sterilized Using Ethylene Oxide
	Do Not Use if Package is Damaged
	Single Use
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician
	For electronic copy scan QR code or go to www.merit.com/ifu and enter IFU ID Number. For printed copy, call U.S.A or E.U Customer Service.
	Medical Device
	Sterile Package
	Manufacturer
	Date of Manufacture: YYYY-MM-DD

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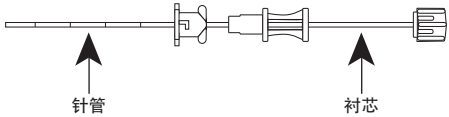
Manufacturer:
Merit Medical Systems, Inc., 1600 West Merit Parkway, South Jordan, Utah 84095 U.S.A.,
1-801-253-1600, U.S.A Customer Service 1-800-356-3748

Authorized Representative:
Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland
EU Customer Service +31 43 358 82 22

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