TEMNO®

Biopsy System

EN	INSTRUCTIONS FOR USE	2
FR	MODE D'EMPLOI	4
IT	ISTRUZIONI PER L'USO	6
DE	GEBRAUCHSANWEISUNG	8
ES	INSTRUCCIONES DE USO	10
PT	INSTRUÇÕES DE UTILIZAÇÃO	12
PTBR	INSTRUÇÕES DE USO	14
NL	GEBRUIKSINSTRUCTIES	16
SV	BRUKSANVISNING	18
DA	BRUGSANVISNING	20
EL	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	22
TR	KULLANMA TALİMATLARI	24
PL	INSTRUKCJA UŻYTKOWANIA	26
CS	NÁVOD K POUŽITÍ	28
BG	ИНСТРУКЦИИ ЗА УПОТРЕБА	30
HU	HASZNÁLATI ÚTMUTATÓ	32
RU	РУКОВОДСТВО ПО ЭКСПЛУАТАЦИИ	34
FI	KÄYTTÖOHJEET	36

TEMNO®

Biopsy System

INSTRUCTIONS FOR USE

DEVICE DESCRIPTION


The TEMNO® biopsy system is a single patient use core biopsy device. It is available in several needle gauge sizes and lengths. The plunger is color coded according to the various gauge sizes (e.g., gray=22G, green=20G, orange=18G, blue=16G, white=14G).

INDICATIONS FOR USE

Intended for use in obtaining core biopsy samples from soft tissue such as breast, kidney, liver, lung, thyroid, and various soft tissue masses. Not intended for use in bone.


STEP 1 COAXIAL TEMNO PLACEMENT

During the pre-procedure set-up, insert the biopsy device through the coaxial introducer needle to verify compatibility. Adjust sliding stopper on coaxial needle to desired calibration level. Lock in place. Insert coaxial and stylet until sliding stopper rests on skin surface. Image coaxial needle to make sure the tip is on the surface of the lesion. Remove inner stylet.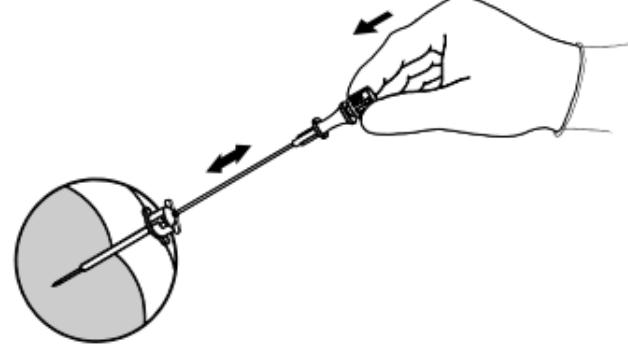


STEP 2 TEMNO NEEDLE SPRING CHARGE

Charge Temno needle by pulling back hard on plunger until a firm click is felt indicating that the needle spring is charged.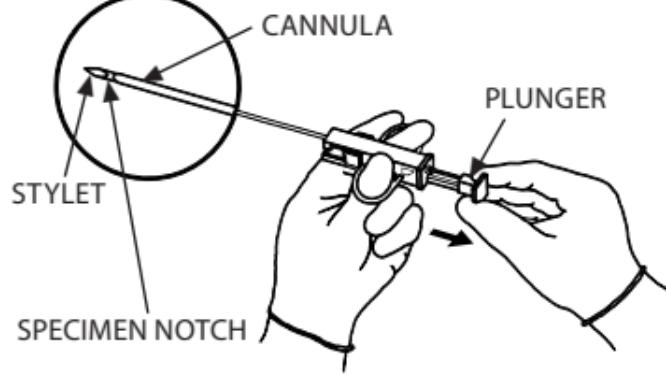


STEP 3 TEMNO NEEDLE PLACEMENT

With stylet fully retracted, (i.e. specimen notch completely covered by the cannula as in STEP 2) insert Temno needle into coaxial and connect coaxial needle hub into luer of Temno. Turn needle 90° to lock.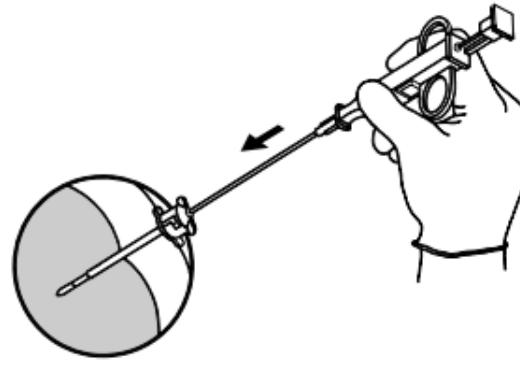

STEP 4 CAPTURE BIOPSY SPECIMEN

Press plunger with thumb to advance specimen notch into area to be sampled. Image again to insure that notch is inside the lesion. Press plunger past stop with thumb, automatically triggering the cannula to close over the notch trapping the specimen.

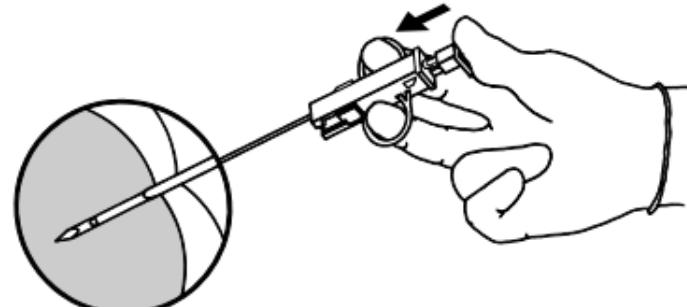STEP 5 RETRIEVE BIOPSY SPECIMEN

Draw needle out. Pull back on plunger to recharge the spring. Push plunger to first stop exposing the specimen notch. Remove tissue specimen from notch in stylet. **Be careful not to push plunger past first stop during this step.**

CAUTION: For single use only. Re-use may result in a non-functional product or contribute to cross contamination.

WARNING:

- Not intended for use in bone.
- Do not use if package is damaged.

Only clinicians who have been properly trained in this procedure, including patient preparation, sample preservation and the use of image guidance for percutaneous biopsy, should use this device.

SYMBOL	DESIGNATION
	Use By: YYYY-MM-DD
	Lot Number
	Catalog Number
	Sterilized Using Ethylene Oxide
	Do Not Use if Package is Damaged
	Single Use
Rx ONLY	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician

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Merit Medical Systems, Inc.
South Jordan, Utah 84095 U.S.A.
U.S.A Customer Service 1-800-356-3748
Merit Medical Ireland Ltd,