SAVI® Preparation Device Instructions for Use

DESCRIPTION
The SAVI Preparation Device (SPC-01 and PREP-01) is a balloon catheter that is inserted into the desired region to be treated at the time of lumpectomy or just prior to placement of the SAVI applicator. This balloon-type catheter acts as a volume indicator for the anatomical site and subsequent cavity evaluation prior to placement of the SAVI applicator. The SAVI Preparation Device is provided sterile and is a single use device.

INDICATIONS
The SAVI Preparation Device is indicated for use as an anatomical tool and aid to placement for the SAVI applicator.
CONTRA-INDICATIONS
Placement in cavities in generally poor condition (e.g., ulcerated) is not recommended with the SAVI Preparation Device.
WARNINGS

The SAVI Preparation Device should only be used by physicians. The physician is responsible for its proper clinical use. After removal, verify no parts remain in the patient.

The SAVI Preparation Device is shipped sterile and MUST NOT BE RESTERILIZED.

The SAVI Preparation Device is for SINGLE USE only.

Do not use if the package is open or damaged.

PLACEMENT OF DEVICE

The SAVI Preparation Device may be placed through a variety of techniques. Always align the long axis of the SAVI Preparation Device to the long axis of the cavity.

1. Remove the balloon protector covering the SAVI Preparation Device.
2. Using the scalpel provided in the SAVI Applicator Kit, create a skin nick.
3. Through the skin nick and using ultrasound guidance, advance the obturator provided in the SAVI Applicator Kit into the cavity.
4. Remove the obturator and under ultrasound guidance advance the SAVI Preparation Device along the same path until the distal tip of the device contacts the far wall of the cavity.
5. Inflate the device with sterile solution to the desired fill volume. Refer to the chart for fluid fill volumes and the corresponding diameter of the device*.

FILL Volume	Approximate Diameter of SAVI Preparation Device
20 cc	3 cm
40 cc	4 cm
60 cc	4.6 cm

*This chart is for reference only. It is not intended to replace clinician discretion.

6. Once the desired diameter has been achieved, withdraw all the fluid from the device.

7. Gently remove the SAVI Preparation Device and discard.

DISPOSAL PROCEDURES

It is the user's responsibility to dispose of all parts according to local regulations. Upon disposal of the product or related parts, it is suggested to:

- Disinfect the parts prior to disposal
- Dispose of sharp objects in a safe manner

STORAGE

Store at room temperature. Avoid storing the SAVI Preparation Device at conditions of excessive heat or humidity.

COMPONENTS

SPC-01	PREP-01		
Description	Quantity	Description	Quantity
Obturador (w/sheath)	1	Obturador	1
Preparation Device	1	Preparation Device	1
Syringe	1	Syringe	1
Scalpel	1		

Symbol	English	Symbol	English
	Instructions for Use Included		Sterilized by Gamma Radiation
	Read Instructions for Use Prior to Use		DO NOT USE IF PACKAGE IS DAMAGED
	Part Number		DO NOT REUSE
	Lot Number		Manufacturer
	USE BY (M/YYYY)		

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