Percutaneous Vessel Dilator

<i>English</i>
Instruction for Use <div>Merit®Medical Percutaneous Vessel Dilator</div>

Description
Merit®Medical percutaneous vessel dilators are intended for single use and are supplied sterile and non-pyrogenic.

Indications
The Merit®Medical vessel dilators are designed for percutaneous entry into a vessel utilizing the Seldinger technique in order to enlarge the opening of the vessel for the placement of a catheter in a vein or artery.

Warnings
Merit®Medical vessel dilators are designed for use with maximum diameter guidewires. The maximum diameter of guidewire to be used is specified on individual vessel dilator labels. The use of guidewires has been associated with greater incidence of thrombus. Optimal guidewire size and judicious use are recommended.

Merit®Medical dilators are designed for one time use only. DO NOT RESTERILIZE. Resterilization could change the physical characteristics of the material and should not be attempted. Variations in individual patient anatomy may preclude the utilization of the percutaneous technique.

Sterilization
Merit® vessel dilators are sterilized with ethylene oxide.

Precautions
The percutaneous technique should be undertaken only by an experienced angiographer.

Vessel dilators are supplied sterile and non-pyrogenic. Do not use if package is opened or damaged.

Employ aseptic technique during removal from the package and during use. Do not autoclave.

To avoid damage and crimping of the dilators during removal from the package, grasp the hub and slowly withdraw from the package.

Care should be taken during placement and withdrawal of dilator in order to prevent possible tissue damage.

Adverse Reactions
Anytime an angiography procedure is performed the possibility exists for arterial wall damage, thrombus formation, emboli formation and plaque dislodgment which could result in myocardial infarction, cardiac arrhythmias, stroke or death. The physician should be familiar with the current literature concerning the complication of angiography.

Directions For Use
The schematic provides the procedure for percutaneous entry of Merit®Medical vessel dilators utilizing the Seldinger technique.

- Puncture vessel with needle.
- Insert flexible (distal end) of guidewire through the needle cannula.
- Remove needle cannula leaving guidewire in position.
- Pass dilator over guidewire to dilate tissue in vessel. Remove dilator and insert catheter over guidewire.
- Remove guidewire leaving catheter in position.

Storage
Handle with care. Do not store in excessive heat. Prolonged exposure to ultraviolet light may cause discoloration and changes in the physical characteristics of the dilator material.

How Supplied
Individually packaged in units of 10 dilators per carton. Also supplied with certain Merit®Medical angiography kits.

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Merit Medical Systems, Inc.
U.S.A. 1-801-253-1600 U.S.A.
Merit Medical Ireland Ltd, Parkmore