

AERO[®]

Tracheobronchial Stent System

English



MERITMEDICAL
ENDOTEK[®]

Patient Implant Card

PLACE THE REMOVABLE STICKER FROM THE PRODUCT LABEL HERE



Merit Medical Systems, Inc.
1600 West Merit Parkway
South Jordan, Utah 84095

www.merit.com/endoscopy



Merit Medical Ireland Ltd
Parkmore Business Park West
Galway, Ireland

MRI SAFETY INFORMATION

Non-clinical testing has demonstrated that the AERO stent is MR Conditional. A patient with this device can be safely scanned in an MR system meeting the following conditions:

- Static magnetic field of 3.0T or less
- Spatial gradient magnetic field of 720 Gauss/cm (7.2 T/m) or less
- Maximum MR system reported, whole body averaged specific absorption rate (SAR) of 3 W/kg for 15 minutes of scanning (Normal Operating Mode)

Under the scan condition defined above, the AERO stent is expected to produce a maximum temperature rise of 0.6° C after 15 minutes of continuous scanning. MR image quality may be compromised if the area of interest is in the exact same area or relatively close to the positions of the AERO stent. Therefore, it may be necessary to optimize MR imaging parameters for the presence of this metallic implant.

Patient Name: _____

Implant Date: _____

Hospital Name: _____

Hospital Address: _____

